Gebrauchsanweisung (DE) | Explorer System | Module & Ring

Seiten 1 bis 41

Table des matières (FR) | Explorer System | Modules & Ring

Pages 42 à 84

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1. Instructions for Use (EN) | Explorer System | Modules & Ring

Valid from: 01.02.2024

- Read this document carefully before using the product.
- Observe the safety instructions to avoid injuries and product damage.
- Instruct the user in the proper and safe use of the product.
- Contact the manufacturer if you have any questions about the product (e.g., commissioning, use, maintenance, unexpected operation or incidents). You will find the contact details on the back.
- Keep this document in a safe place.

These instructions for use provide you with important information on the use, adjustment and handling of modules and additional accessories of the Explorer System.

2. Use

The macu4 Explorer Modules are intended to be used as passive functional hands that serve as accessories to a custom-made or patient matched socket prosthesis, which is intended for the prosthetic replacement of the forearm.

Product description

The Modules are accessories to the Explorer Socket and are to be configured with a spacer, depending on the indication and the difference in length between the healthy arm and the arm stump.


The Explorer Ring is used to make it easier to position the Explorer Module used in the desired orientation. The Explorer Ring is used together with any Module from the Explorer System that serves as an accessory of a medical device.

Overview on Modules and additional accessories:

- BALL Module: Support for certain ball sports, e.g., to pick up a ball from the ground, to hold and/or throw a ball.
- BIKE Module: Support for cycling on public paths and roads, e.g., to maintain balance while cycling (for users who can cycle without a prosthesis).
- PINCH Module: Support for various activities with flat or cylindrical objects, e.g. playing cards, credit cards, paper, pens, cutlery, etc.
- SWIM Module: Support for specific swimming styles and strokes.
- TWIN Module: Support for various activities with a bar or bars.
- RING STD: To be used with the Explorer Socket
- RING EXP: To be used with a prosthetic socket with flat distal end

Indication

The Explorer Modules can be used for unilateral amputees from amputation height transradially or with dysmelia in forearm fittings (according to ISO 8548-2:2020) as well as for categories 2 to 5, as shown in the diagram below.

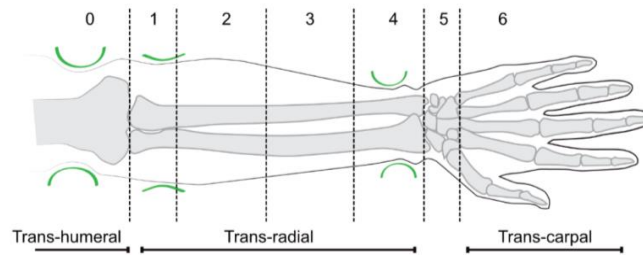The system is suitable for persons 2 years and older. The Pinch module is suitable for persons 4 years and older.

Contraindications

- The Explorer System is not indicated for individuals with bilateral forearm deficits.
- The Explorer System is not indicated for individuals who have musculoskeletal limitations (e.g., stiffness in the shoulder joint).
- The system may only be used by users who are physically and mentally fit to operate the system properly. If you are unsure whether the physical and/or mental fitness is sufficient, we recommend an individual assessment by the specialist (e.g. doctor, therapist, prosthetist).

Combination possibilities

The Explorer Modules have been tested for combination possibilities with the Explorer socket. See chapter 10 for more information on the load-bearing capacity of the Modules. For other combination possibilities, the specialist must check on his own responsibility whether the combination with the Explorer Modules is possible.

Operating conditions

The Explorer System was developed especially for everyday, sports and leisure activities with low to moderate stress. The system must not be used for extraordinary activities. These extraordinary activities include, for example, sports with excessive coupling and/or impact loads (push-ups, downhill and mountain biking, etc.) or extreme sports (free climbing, paragliding, etc.). In addition, the system should not be used for driving motor vehicles, driving heavy equipment (e.g., construction machinery), operating industrial machinery or operating motor-driven work equipment.

3. Warnings

Notes on unpacking the supplied components

Only open the packaging at the intended points with care so that the parts inside are not damaged. Inspect the delivered components and make sure that they are undamaged. If the components are damaged, please follow the steps in chapter 12.

Notes on commissioning the Modules

Please familiarise yourself with the use of each Module before using it for the first time.

Instructions for checking before use

Inspect the Modules before each use and make sure that they do not show any visible damage. Improper handling of a Module can restrict its function or render it inoperable.

Please also observe the information on the mechanical load capacity of a Module. The mechanical stability of the Module may be impaired if it is used and/or stored outside the permissible temperature range. If possible, UV radiation should be avoided during use, but absolutely prevented during storage so as not to accelerate the aging process.

Notes on using the Explorer RING together with a Module

Make sure that you always use the Explorer RING EXP together with the Explorer Modules when you use the Modules together with the Explorer socket. The function of the Explorer RING is to make it easier for you to bring and secure a Module into the desired orientation.

Notes on Modules with and without a spacer

A Module is delivered to you in assembled condition. A Module must not be disassembled independently. This also applies to Modules that are equipped with a spacer. In this case, the spacer is an integral part of the Module. The spacer must not be disassembled. Make sure that you do not manipulate Modules (e.g., by removing product components).

Notes on coupling a Module to the socket

Make sure that you always couple a Module completely to the socket.

Notes on disconnecting a Module from the socket

Make sure that you always disconnect a Module completely from the socket.

Notes on the use of the BIKE

Make sure that the BIKE has a good enough grip on the handlebars before using it for the desired activity.

Notes on the use of the PINCH

Make sure that the lever of the ball joint is tightened and that the Module fingers clamp the object well enough and do not come loose unintentionally.

Notes on the use of the TWIN

Make sure that the helix of the TWIN encloses the rod properly and cannot come loose unintentionally.

Notes on the load capacity of the Modules

Make sure that you do not overload the Modules.

Notes on the care of the socket

Make sure that you always clean the Modules after use. Remove visible dirt particles so that Modules with flexible modes are not restricted in their function.

Notes on ambient conditions


Make sure that you store or use the products in accordance with the information on temperature, UV radiation and humidity given in chapter “Technical Data”.

Notes on magnets in the product


The RING contains mini magnets (5x5x5mm). Make sure that you keep the socket Ring at least 5cm away from e.g. pacemakers or defibrillators.

4. Use of the Ring


Before using the Explorer Ring for the first time, familiarise yourself with the instructions for use. The chapter “Use of the Modules” explains how to use the RING together with the Explorer Modules.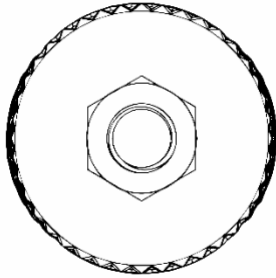

RING – Top View. This side of the RING faces towards the Module (distal side).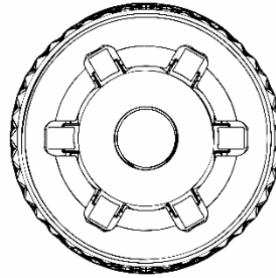

RING – Bottom View. This side of the RING faces towards the Socket (proximal side).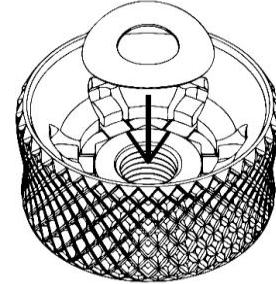


Metal washer. It is connected to the RING via magnets. If the washer gets lost, use the replacement part.

5. Use of the Modules


Before using the Explorer Modules for the first time, familiarise yourself with the instructions for use. The following chapters explain how to use the Modules together with the Explorer socket.

5.1. BALL Module

Explorer BALL Module and socket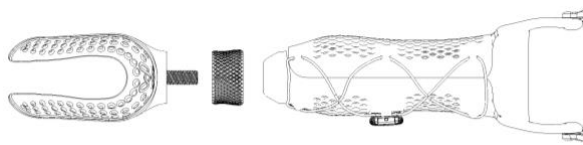


Module without spacer

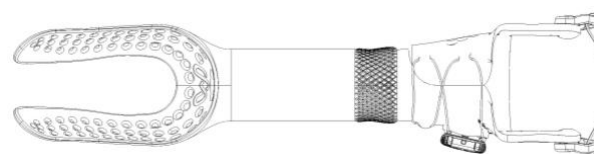
Module with spacer (here with medium spacer length)

Module design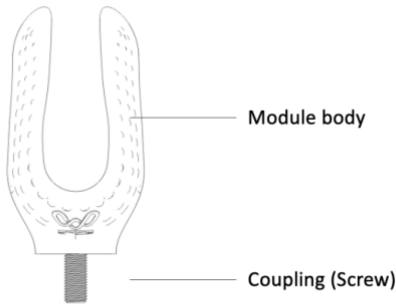


Module design without spacer

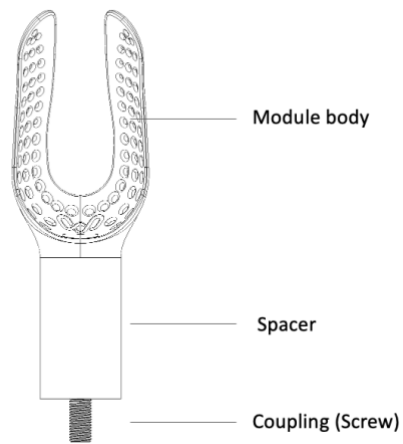Module design with spacer

Couple the Module to the socket

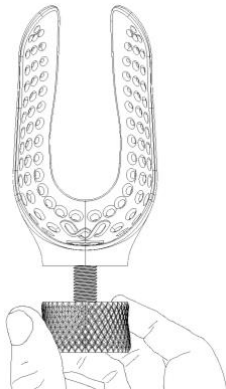Screw the socket ring onto the Module as far as possible, but do not tighten it.

Insert the screw end of the Module into the socket.


Screw the Module tight with the socket ring until you have achieved the desired alignment.

Tighten the ring to secure the position.

Move the Module to the desired orientation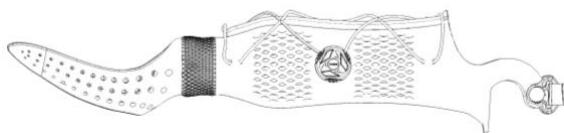


The socket ring system offers the possibility to freely select the orientation of the BALL Module.

The standard alignment can be seen in the illustration.


For different activities, the alignment of the Module with the socket can be adjusted, deviating from the standard alignment.

Final control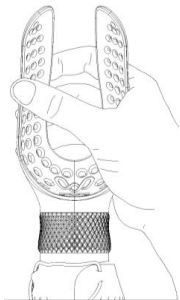


- Check the Module before you start any activity.
- Make sure it is well positioned and secured by trying to move the Module gently.
- Note that the Module should remain firmly in position.
- If the Module is not tight and can be moved, please tighten the socket ring again.
- Be careful not to unlock the system during the check.


Disconnect the Module from the socket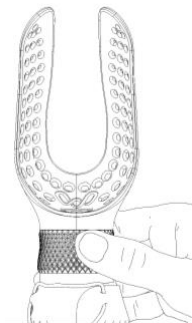

Loosen the socket ring by turning it clockwise as seen from the socket.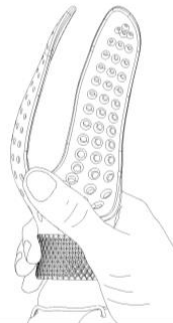


Unscrew the Module with the socket ring from the socket.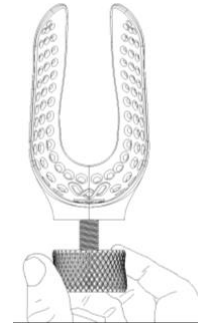


Finally, unscrew the socket ring from the Module by turning it to the left.

5.2. BIKE Module

Explorer BIKE Module and socket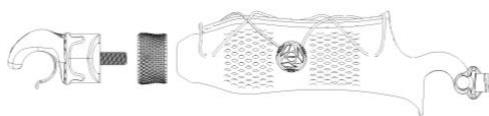


Module without spacer

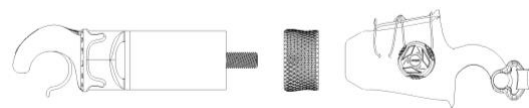
Module with spacer (here with medium spacer length)

Module design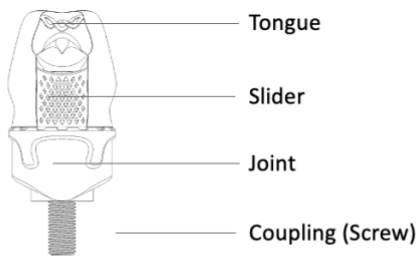


Module design without spacer

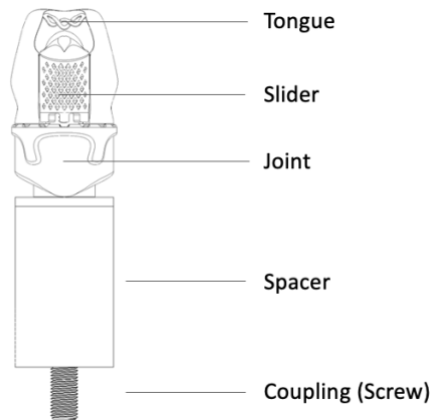
Module design with spacer

Couple the Module to the socket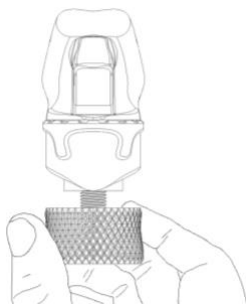


Screw the socket ring onto the Module as far as possible, but do not tighten it.

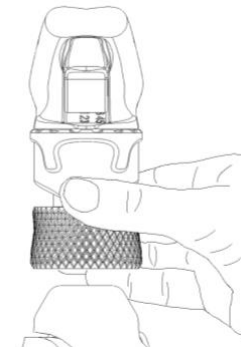Insert the screw end of the Module into the socket.


Screw the Module tight with the socket ring until you have achieved the desired alignment.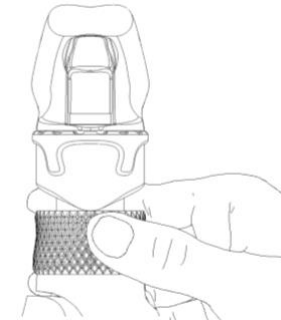


Tighten the ring to secure the position.

Move the Module to the desired orientation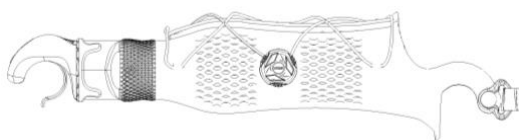


The socket ring system offers the option of freely selecting the orientation of the BIKE Module. The standard alignment can be seen in the illustration. For different activities, the alignment of the Module with the socket can be adjusted, deviating from the standard alignment.


Use of the slider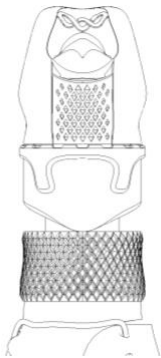


Use the slider to attach the Module to vertically aligned handlebars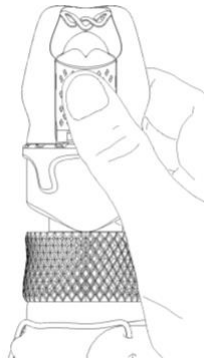

Unlock the Module by sliding the slider towards the tongue.

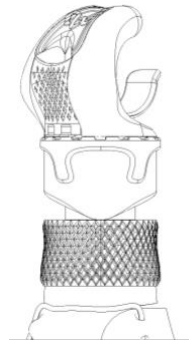
Turn the Module to the right (on the right arm) or to the left (on the left arm) until the desired alignment is reached.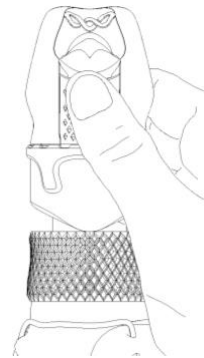

Block the Module by pushing the slider towards the socket.

Final control

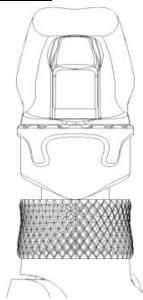
- Check the Module before you start any activity.
- Make sure it is well positioned and secured by trying to move the Module gently.
- Note that the Module should remain firmly in position.
- If the Module is not tight and can be moved, please tighten the socket ring again.
- Be careful not to unlock the system during the check.


Disconnect the Module from the socket


Make sure that the slider blocks the Module.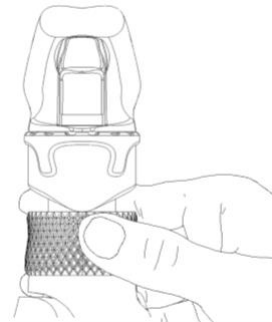

Loosen the stem ring by turning it clockwise as seen from the stem.

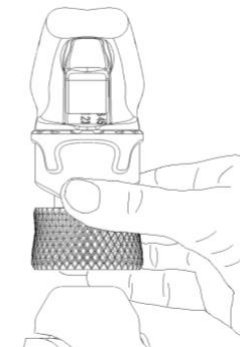
Unscrew the Module with the socket ring from the socket.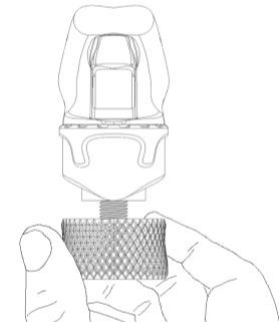

Finally, unscrew the socket ring from the Module by turning it to the left.

Replacement of Module Tongue

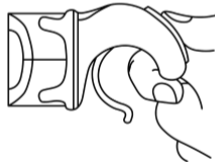Release the Module tongue from the inside by applying pressure.


Remove the Module tongue from its anchorage.


Insert the other Module tongue and click it into place by applying slight pressure.

5.3. PINCH Module

Explorer PINCH Module and socket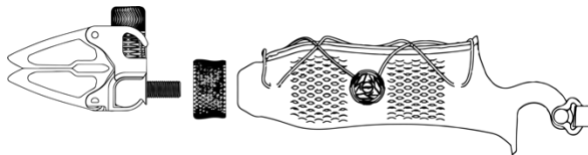


Module without spacer

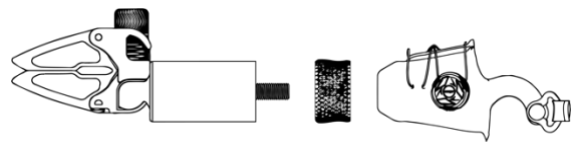
Module with spacer
(here with medium spacer length)

Module design

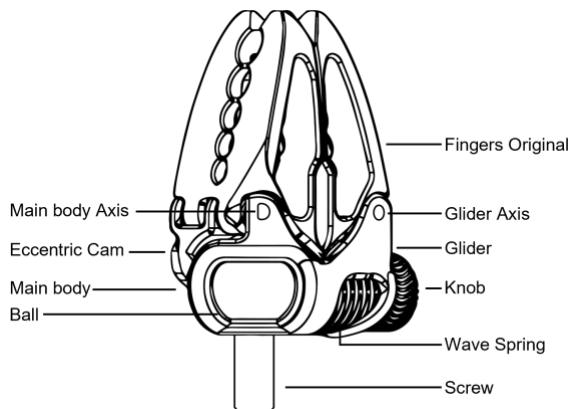


Module structure without spacer

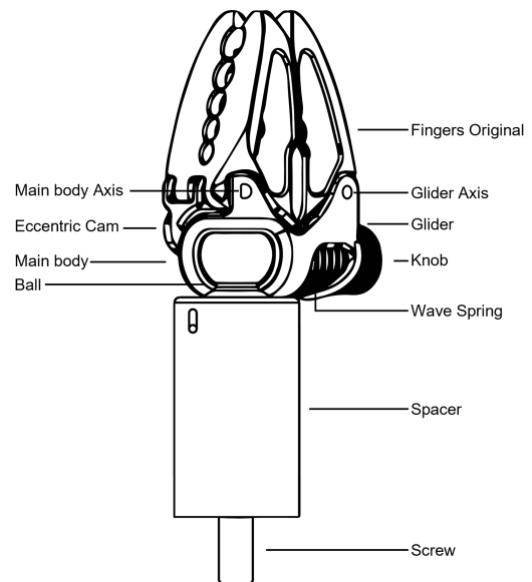Module structure with spacer




Coupling the Module to the socket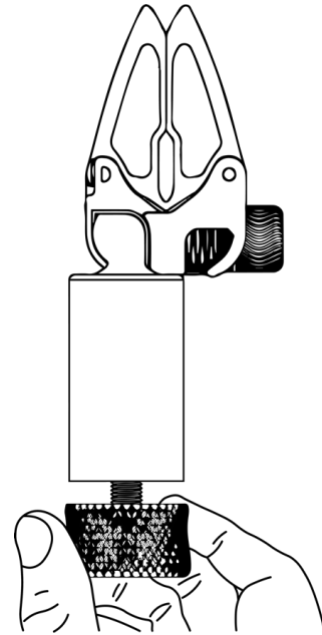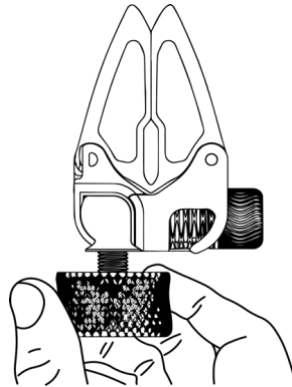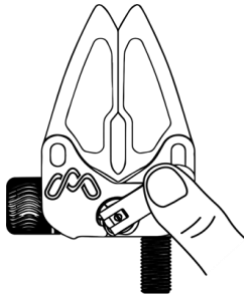

Note: Before coupling the Module to the socket, make sure that the ball joint is locked with the lever.

Without spacer: Screw the socket ring onto the Module as far as possible, but do not tighten it.

With spacer: Screw the socket ring onto the Module as far as possible, but do not tighten it.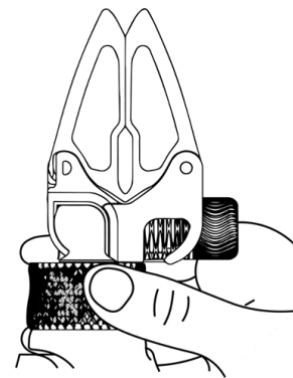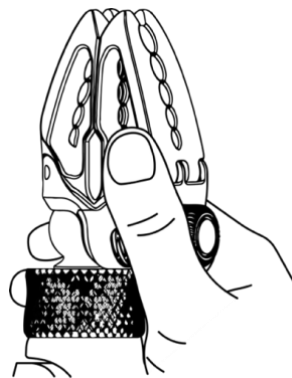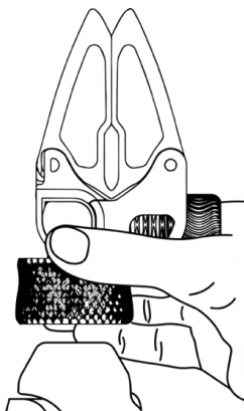


Insert the screw end of the Module into the socket.

Screw the Module together with the socket ring into the socket as far as it will go.


Tighten the socket ring to secure the position.

Move the Module to the desired orientation


When the socket ring is tightened, and the ball joint of the Module is locked, it creates a rigid connection with the socket. You can adjust the module orientation whether the fingers are open or closed.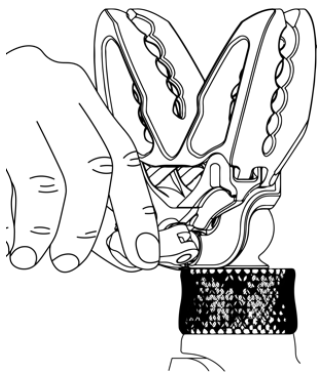

To bring the Module into the desired position, loosen the lever of the ball joint of the Module.

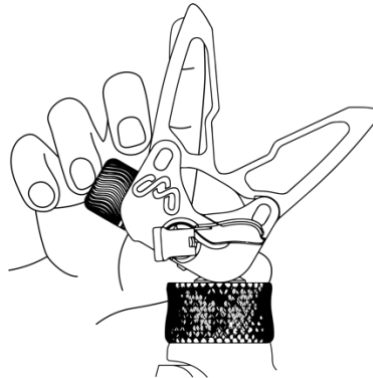
Rotate the Module with your hand to reach the desired orientation.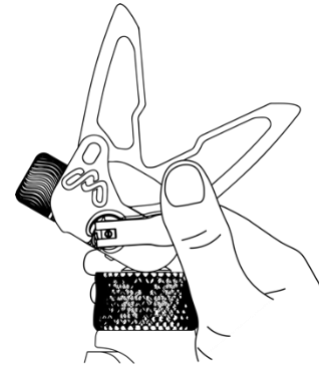

When the desired orientation is reached, tighten the lever.

Final control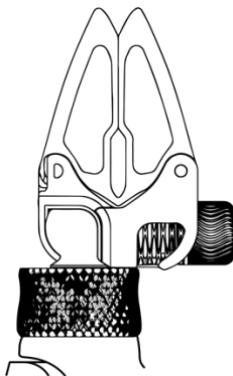


- Check the Module before starting any activity.
- Make sure it is well positioned and secured by trying to move the Module gently.
- Note that the Module should remain firmly in position.
- If the Module is not tight and can be moved, please retighten the socket ring and the lever (see "Adjusting the rigidity of the ball joint" on the next pages).
- Be careful not to unlock the system during the check.

Fixation of objects in the Module body

- To open the Module, move the two tips of the Module fingers apart with your hand.
- Place a flat object (e.g. playing cards) or a medium-sized cylindrical object between the Module fingers and close the Module fingers with your hand.
- Depending on the geometry of the object, it can also be pushed between the closed Module fingers.
- To remove the item, you can either pull it until it is free or you can open the Module fingers by hand.
- Small-sized cylindrical object objects with different circumferences can also be pressed down into the slot with the round holes from above until they engage.

Use Module with a flat or medium-sized cylindrical object

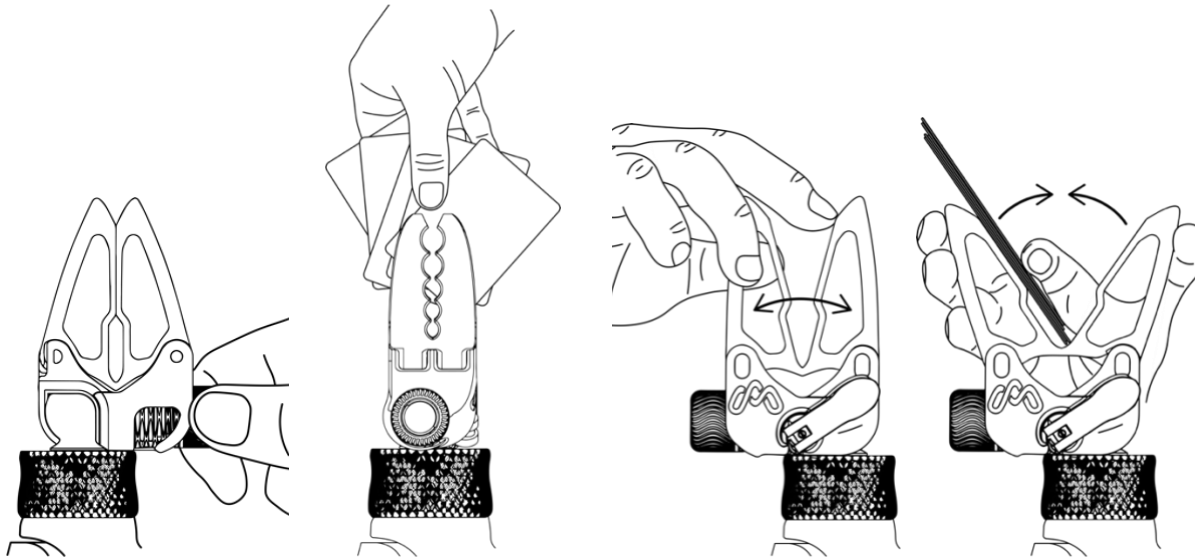Close the Module fingers and adjust the gripping force with the knob.

Carefully slide the cards between the Module fingers.

Alternatively, you can open the Module fingers first by moving the tips of the Module fingers apart.

Then position the cards between the Module fingers and close them by squeezing them with your hand.

Note: If the Module fingers cannot be closed (pressed against each other), adjust the gripping force of the Module fingers with the knob until it is satisfying.

Use Module with a small-sized cylindrical object

Close the Module fingers and adjust the gripping force with the knob. In this case, no firm grip is required.

Using your hand, press the pen down from above between the holes on the side of the Module fingers until the pen clicks into place.


Note: Do not push the cylindrical object too far. It could cause irreversible damage to the Module fingers. The maximum diameter accepted, as small-sized, with this approach is provided in the "Technical data".

You can also hold larger cylindrical objects by following the same steps as for flat objects. The best approach varies from object to object and is left to your own judgement.

Correct the alignment of the Module

Follow the steps mentioned under “Move the Module to the desired orientation”.

Adjusting the rigidity of the ball joint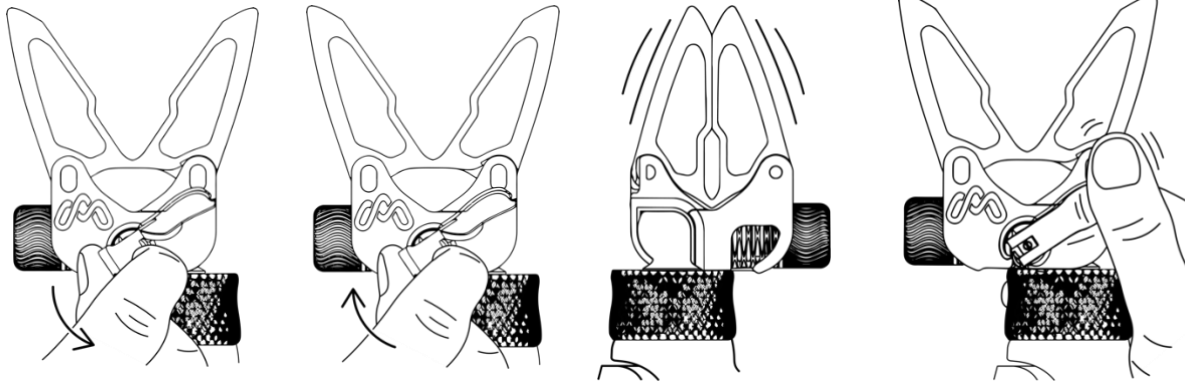


By opening the lever and turning it anticlockwise, you can reduce the rigidity of the ball joint.

By opening the lever and turning it clockwise, you can increase the rigidity of the ball joint.

If the alignment changes unintentionally even though the ball joint is locked, increase the rigidity of the ball joint.

If closing the lever is too difficult, reduce the rigidity of the ball joint by rotating the lever anticlockwise.

Adjustment of the gripping force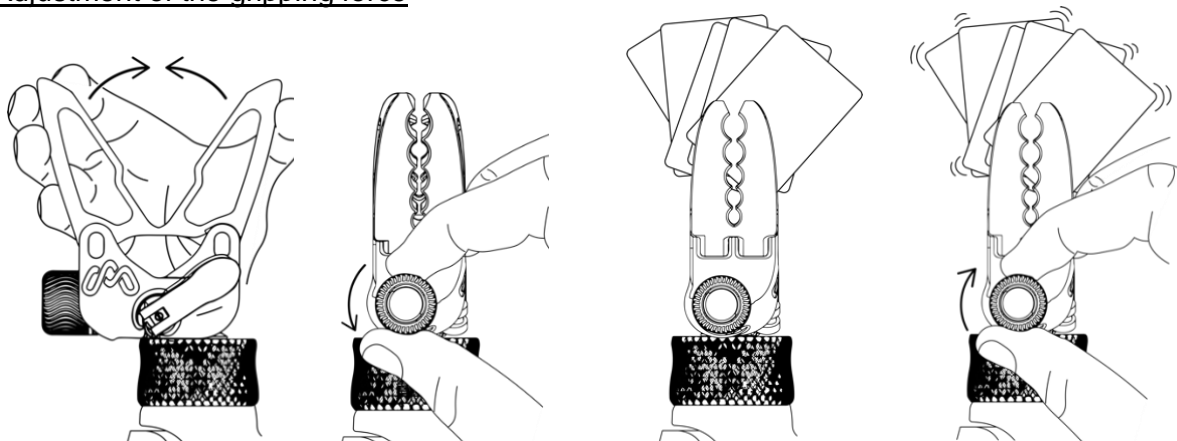


Close the Module fingers with your hand.

If closing is too difficult, release the tension in the Module fingers by turning the knob anticlockwise.


Try to grasp a flat object (e.g. playing cards) with the fingers.

If the gripping force is not sufficient, apply more tension to the Module fingers by turning the knob clockwise.


Disconnect the Module from the socket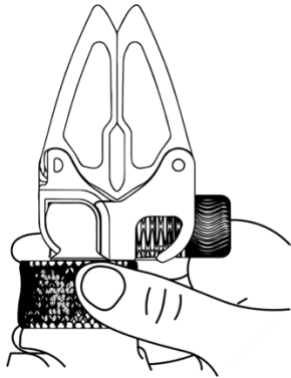

Loosen the socket ring by turning it clockwise as seen from the socket.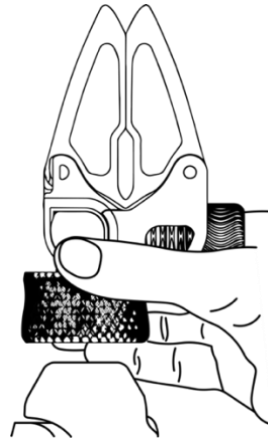


Unscrew the Module with the socket ring from the socket.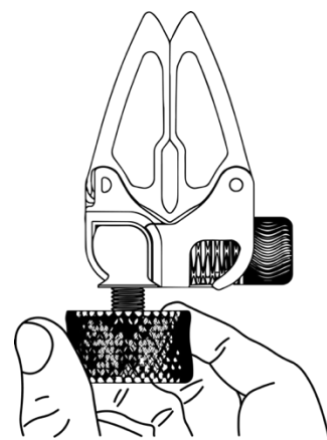

Finally, unscrew the socket ring from the Module by turning it to the left.


5.4. SWIM Module

Explorer SWIM Module and socket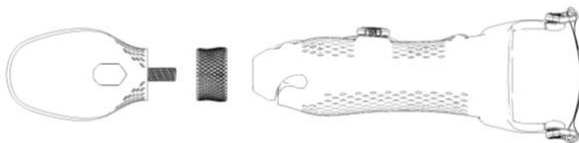


Module without spacer

Module design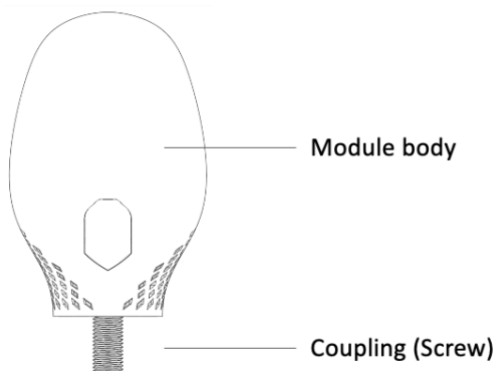

Module design without spacer

Couple the Module to the socket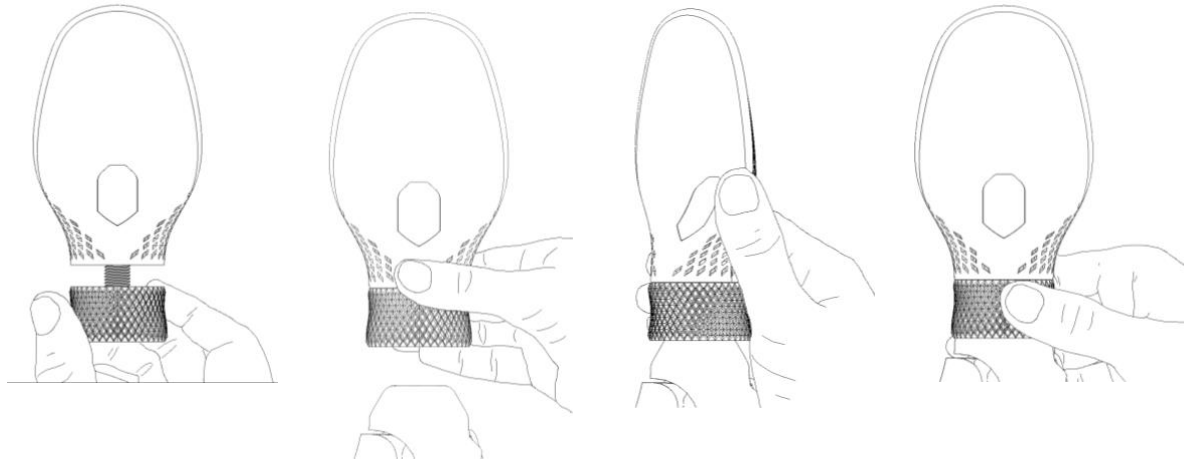


Screw the socket ring onto the Module as far as possible, but do not tighten it.

Insert the screw end of the Module into the socket.

Screw the Module tight with the socket ring until you have achieved the desired alignment.

Tighten the ring to secure the position.

Move the Module to the desired orientation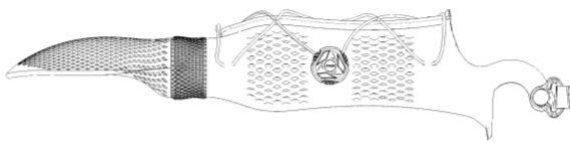


The socket ring system offers the possibility to freely select the orientation of the SWIM Module.

The standard alignment can be seen in the illustration.


For different activities, the alignment of the Module with the socket can be adjusted, deviating from the standard alignment.

Final control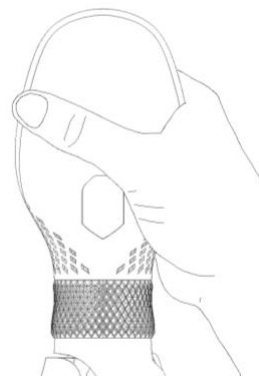


- Check the Module before you start any activity.
- Make sure it is well positioned and secured by trying to move the Module gently.
- Note that the Module should remain firmly in position.
- If the Module is not tight and can be moved, please tighten the socket ring again.
- Be careful not to unlock the system during the check.


Disconnect the Module from the socket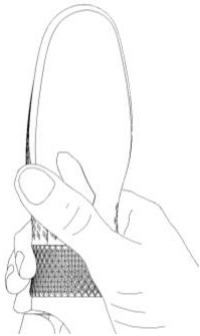

Loosen the socket ring by turning it clockwise as seen from the socket.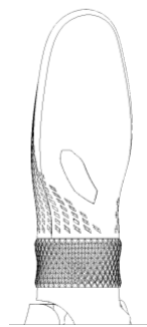


Unscrew the Module with the socket ring from the socket.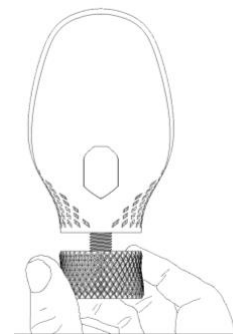


Finally, unscrew the socket ring from the Module by turning it to the left.

5.5. TWIN Module

Explorer TWIN Module and socket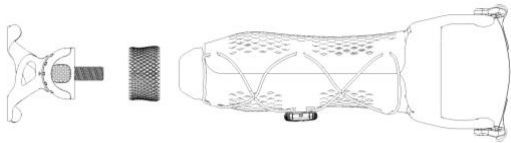


Module without spacer

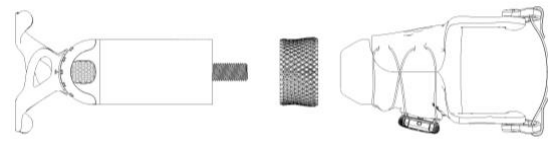
Module with spacer (here with medium spacer length)

Module design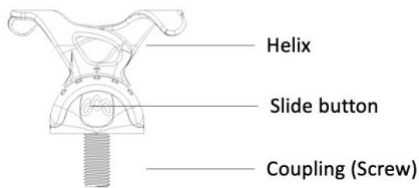

Module design without spacer

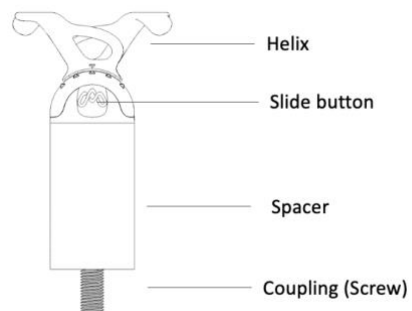
Module design with spacer


Couple the Module to the socket

Without spacer: Screw the socket ring onto the Module as far as possible, but do not tighten it.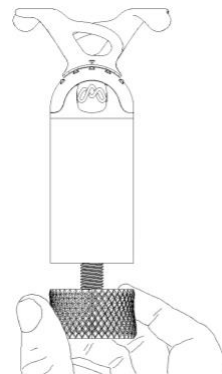


With spacer: Screw the socket ring onto the Module as far as possible, but do not tighten it.

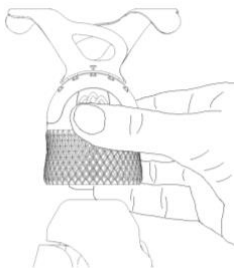Insert the screw end of the Module into the socket.


Screw the Module tight with the socket ring until you have achieved the desired alignment.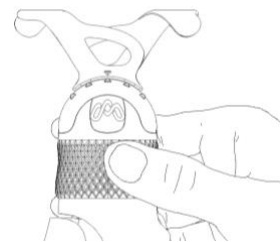


Tighten the ring to secure the position.

Move the Module to the desired orientation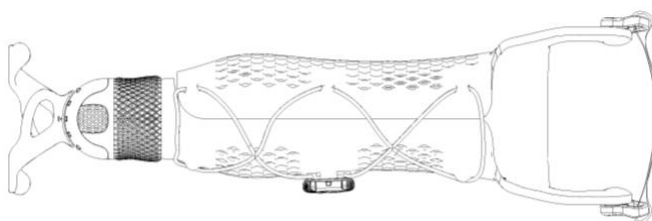


The socket ring system offers the possibility of freely selecting the orientation of the TWIN Module. The standard alignment can be seen in the illustration. For different activities, the alignment of the Module with the socket can be adjusted, deviating from the standard alignment.

Final control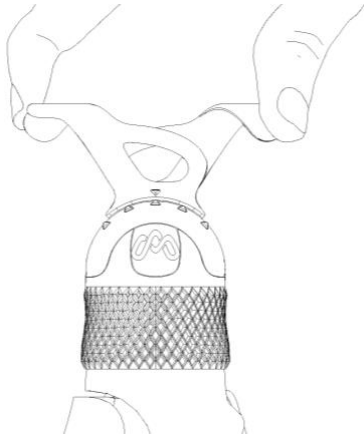


- Check the Module before you start any activity.
- Make sure it is well positioned and secured by trying to move the Module gently.
- Note that the Module should remain firmly in position.
- If the Module is not tight and can be moved, please tighten the socket ring again.
- Be careful not to unlock the system during the check.


Click Module to a rod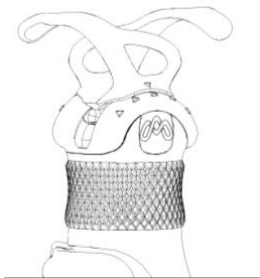


Block the Module using the sliding knob.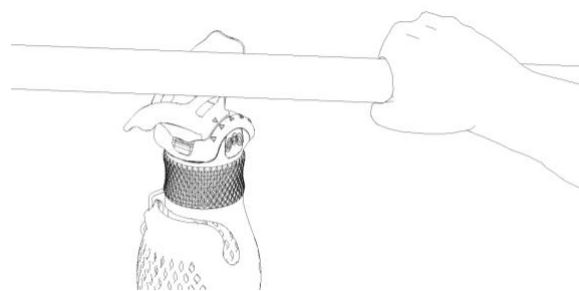

Position the rod across the Module.

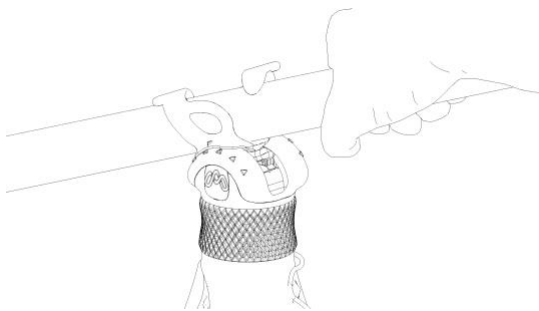
Press the Module onto the rod with the help of your hand so that the helix grippers enclose the rod.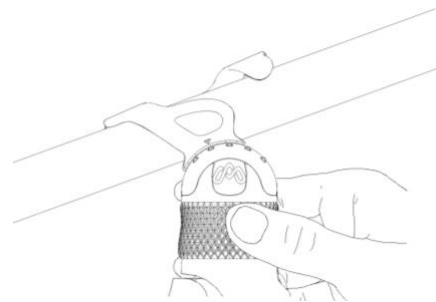

If necessary, correct the alignment of the Module using the socket ring.

Use of the Module joint

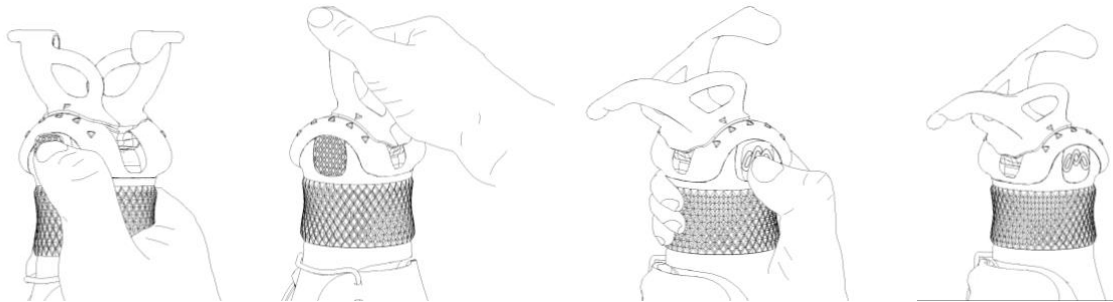Press the slide button to unlock the Module.

Set the desired orientation of the Module.

Lock the system by pushing the sliding knob back to its original position.

End position Note: for some activities, the Module can also remain unlocked.

Disconnect the Module from the socket

Loosen the socket ring by turning it clockwise as seen from the socket.

Unscrew the Module with the socket ring from the socket.

Finally, unscrew the socket ring from the Module by turning it to the left.

6. Module maintenance

Dirt particles in the spaces between the products should be removed with a soft-bristle brush. This pre-cleaning of the products should be done without the use of water. Product surfaces should be cleaned with a damp cloth (non-scratching). Cold or lukewarm water should be used for this purpose.

7. Adverse event reporting

Any serious incident related to the device should be reported to the manufacturer and the competent authority of the member state where the user and/or patient is established.

Please use the form Please use the Product Experience Report (PER)

https://25462115.fs1.hubspotusercontent-eu1.net/hubfs/25462115/mac4_AG_Explorer/Product_Experience_Report_EN.pdf and

send it to by email to support@macu4.com.

8. Legal notice

Limitation or exclusion of liability

macu4 AG, hereinafter referred to as Macu, is only liable if the specified handling and processing instructions as well as the care instructions and maintenance intervals for the product are observed. Macu expressly points out that this product may only be used in the component combinations approved by the manufacturer (see instructions for use and catalogues). The manufacturer is not liable for damage caused by component combinations and applications that have not been approved by the manufacturer. Opening and repairing of the products may only be carried out by Macu or with the written permission of Macu.

9. Conformity

The product components of the system Explorer with the Basic UDI-DI marking

- 76499889892P01RINGHU
- 76499883732P01BALL27
- 76499883732P01BIKE2W
- 76499889892PINCH4N
- 76499889892P01SWIML6
- 76499883732P01TWIN9E

meet the general safety and performance requirements according to Regulation (EU)2017/745 for medical devices and the Medical Devices Ordinance (MedDO) of 1 July 2020 (812.213).

Legal manufacturer

macu4 AG, Rämistrasse 18, 8001 Zurich, Switzerland, CHE-301.889.074

CE Marking Details

Macu declares that they comply with the relevant European standards for the design, manufacture and supply of prosthetic products.

The Accessories (Modules and Ring) of the Explorer System meet the requirements of the European Regulation (EU)2017/745 Annex I for medical devices. Based on the classification criteria according to Annex VIII of this regulation, the product has been classified as Class I “Accessories”. The declaration of conformity has, therefore, been drawn up by the manufacturer under his sole responsibility in accordance with Annex IV of the Regulation. All individual Modules of the Explorer system are marked accordingly.

Trademark

All designations mentioned in this document are subject without restriction to the provisions of the applicable trademark law and the rights of the respective owners.

All trademarks, trade names, or company names mentioned herein may be registered trademarks and are subject to the rights of their respective owners.

The absence of explicit identification of the trademarks used in this document does not imply that a designation is free of third-party rights.



10. Symbols

The table below shows all symbols (according to ISO 15223-1:2021) used on the product and on the label.


CE Mark	
	This mark indicates that a manufacturer declares that the product meets all the legal requirements for CE marking and can be sold throughout the EEA.
Consult operating instructions or electronic instructions for use	
	This symbol indicates that the user must consult the instructions for use.
Manufacturer	
	This symbol indicates the manufacturer of the medical device.
Warnings	
	This symbol indicates that caution is required when operating the product or that the current situation requires the operator's attention or action to avoid undesirable consequences.
Temperature limit	
	This symbol indicates the temperature limits to which the medical device can be safely exposed.
Date of manufacture	
	This mark indicates when the medical device was manufactured.
Batch designation	
	This mark indicates the manufacturer's batch code so that the batch or lot can be identified.
Catalog number	
	This mark indicates the manufacturer's catalogue number so that the item can be identified.
Medical device	
	Indicates that the item is a medical device.
Clear device identification	
	Identifies a carrier that contains information for unique device identification.
Keep away from sunlight	
	This symbol indicates a medical device that must be protected from light sources.
Magnetic fields	
	This symbol indicates that magnets are contained in the product. The magnetic field of magnets may affect or disable the function of medical devices (e.g. pacemakers or defibrillators).


11. Technical data

11.1. RING

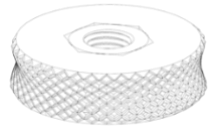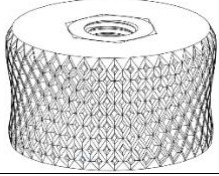General	RING STD	RING EXP
Drawing		
Dimensions	L: 47mm, B: 46mm, H: 13mm	L: 47mm, B: 46mm, H: 26mm
Height of nut	Metric (M12x1.5): 10mm Inch (1/2"-20): 10mm	
Weight	36g	43g
Lifetime of the Ring	2 years at an average of 120 minutes per day with daily use	
Mini magnets		
Specifications	5 x 5 x 5mm, Neodymium N42	
Maximum load forces on screw interface		
Power	approx. 250 N	

11.2. BALL Module

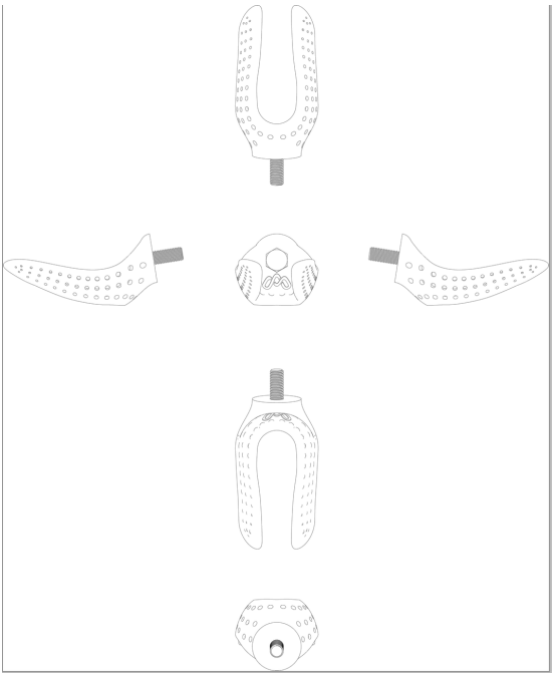
General	
Drawing	

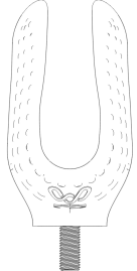	Size 'Standard':	Size 'Small':
		
Dimensions size 'Standard'	L: 161 mm (with screw) L: 134 mm (without screw) B: 75 mm T: 60 mm	
Dimensions size 'Small'	L: 131 mm (with screw) L: 107 mm (without screw) B: 74 mm T: 60 mm	
Length of screw (from lower surface of Module)	Screw, Metric (M12x1.5): 24 mm Screw, inch (1/2"-20): 25.75mm	
Weight	Size 'Standard': 87 g Size 'Small': 85 g	
Alignment of the Module	Module inner side points anteriorly	
Lifetime of the Module	2 years at an average of 120 minutes per day with daily use	
Maximum load forces in the palm of the hand		
Weight of objects (e.g. a ball)	0.2 kg / 0.44 lbs	
Force "throw up" (e.g. tennis ball)	Approximately 15 N	
Force "catching" (e.g. tennis ball)	Approximately 15 N	
Maximum load forces in the "coupling" of macu4		
Axial tensile force	Approximately 50 N	

11.3. BIKE Module

General	
Drawing	
Dimensions	L: 108 mm (with screw) L: 84 mm (without screw) B: 48 mm T: 54 mm
Supported handlebar circumferences	<div style="display: flex; justify-content: space-around;"> <div style="text-align: center;"> <p>Tongue Standard: Ø between 78 mm and 100 mm</p> </div> <div style="text-align: center;"> <p>Tongue Large: Ø between 100 mm and 113 mm</p> </div> </div>
Supported circumferences of handlebars	between 78 mm and 100 mm
Length of screw (from lower edge of Module)	Screw, Metric (M12x1.5): 24 mm Screw, inch (1/2"-20): 25.75mm
Weight (without spacer)	109 g
Abduction of the wrist	25°
Wrist adduction	25°
Alignment of the Module	The inside of the Module faces the handlebar grip
Lifetime of the Module	2 years at an average of 120 minutes per day / daily use
Maximum load forces in the palm of the hand	
Force normal to socket axis (e.g. support on handlebars)	Approximately 55 N
Maximum load forces in the "coupling" of macu4	
Axial tensile force (e.g. driving uphill)	Approximately 250 N

11.4. PINCH Module

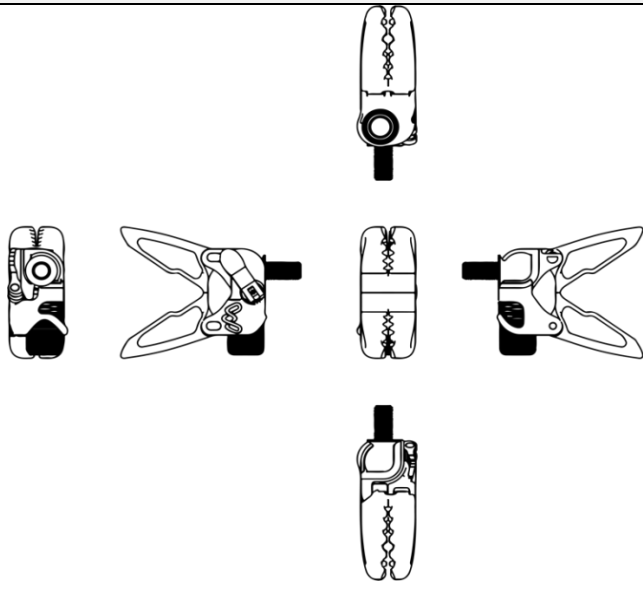General	
Drawing	
Dimensions	Standard size, open Fingers: L: 124 mm (with screw) L: 99.5 mm (without screw) B: 100 mm T: 41 mm Standard size, closed Fingers: L: 125 mm (with screw) L: 100.5 mm (without screw) B: 75 mm T: 41 mm
Maximum diameter for small-sized cylindrical objects	Ø: 12 mm
Maximum diameter for medium- sized cylindrical objects	Ø: 25 mm
Maximum thickness for flat objects	Thickness: 20 mm
Length of screw (from lower edge of Module)	Screw, Metric (M12x1.5): 24 mm Screw, inch (1/2"-20): 24.5 mm
Weight (without spacer)	164 g
Lifetime of the Module	2 years at an average of 120 minutes per day with daily use
Maximum load forces between the Module fingers	
Maximum mass that can be held by the fingers (e.g. holding a book)	Approximately 1 kg* *may vary based on the geometry and material of the objects
Maximum load forces in the "coupling" of macu4	
Axial tensile force	Approximately 50 N

Note on storage: Always open the lever and the Module fingers before putting the Module away so that the tension is taken off the fingers.








11.5. SWIM Module


General				
Drawing				
	<table border="1" style="width: 100%; text-align: center;"> <tr> <td style="width: 50%;">Standard size:</td> <td style="width: 50%;">Small size:</td> </tr> <tr> <td></td> <td></td> </tr> </table>	Standard size:	Small size:	
Standard size:	Small size:			
Dimensions	<p>Standard size: L: 147 mm (with screw) L: 123 mm (without screw) B: 80 mm T: 45 mm</p> <p>Small size: L: 112 mm (with screw) L: 88 mm (without screw) B: 65 mm T: 44 mm</p>			
Length of screw (from lower edge of Module)	Screw, Metric (M12x1.5): 24 mm Screw, inch (1/2"-20): 25.75mm			
Weight	SWIM Standard: 90 g SWIM Small: 75 g			
Alignment of the Module	Module inner side points anteriorly			
Lifetime of the Module	2 years at an average of 120 minutes per day with daily use			

Maximum load forces in the palm of the hand

Force to socket axis (e.g. pulling Module through water)	Approximately 35 N
Maximum load forces in the "coupling" of macu4	
Axial tensile force	Approximately 50 N

11.6. TWIN Module

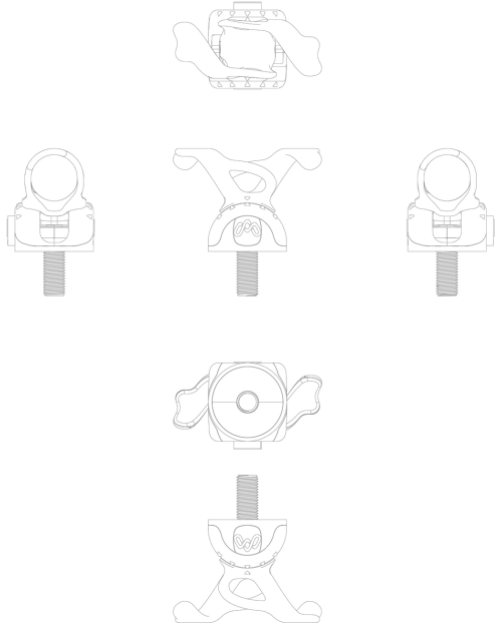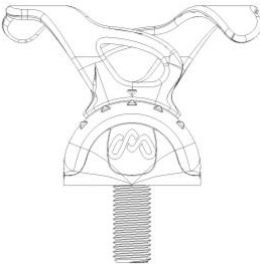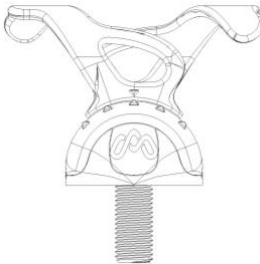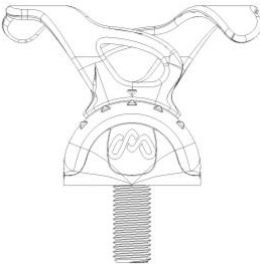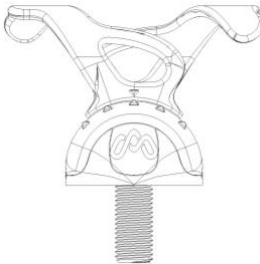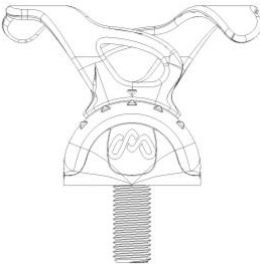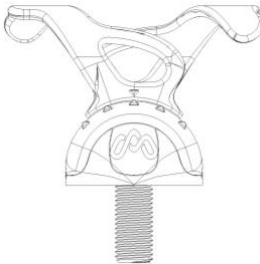General					
Drawing					
	<table border="1"> <tr> <td>Standard size:</td> <td>Big size:</td> </tr> <tr> <td>  </td> <td>  </td> </tr> </table>	Standard size:	Big size:		
	Standard size:	Big size:			
					
Dimensions size 'Standard'	L: 83 mm (incl. screw neck) L: 58 mm (without screw neck) B: 83 mm T: 49 mm				
Dimensions size 'Large'	L: 95 mm (incl. screw neck) L: 70 mm (without screw neck) B: 120 mm T: 49 mm				

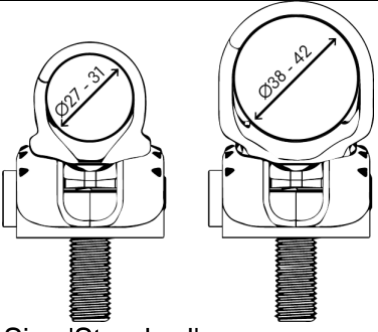Supported diameters of rods	 <p>Size 'Standard': Ø: from 27 mm to 31 mm 28 mm optimal for rubberised bars 31 mm optimal for smooth bars</p> <p>Size 'Large' Ø: from 38 mm to 42 mm 39 mm optimal for rubberised rods 42 mm optimal for smooth rods</p>
Length of screw (from lower edge of Module)	Screw, Metric (M12x1.5): 24 mm Screw, inch (1/2"-20): 25.75 mm
Weight (without spacer)	Size 'Standard': 86 g Size 'Large': 96 g
Number of positions to be adjusted	5
Alignment of the Module	Depending on the activity
Lifetime of the Module	2 years at an average of 120 minutes per day with daily use
Maximum load forces in the palm of the hand	
Weight of objects (e.g. rod)	10 kg / 22 lbs (without jerky movements)
Force "tensile" (e.g. a kayak paddle in the water)	Approximately 110 N
Maximum load forces in the "coupling" of macu4	
Axial tensile force	Approximately 110 N

11.7. Environmental conditions

Environmental conditions	
Storage and transport in the original packaging	0 °C/32 °F to +35 °C/95 °F max. 70 % relative humidity
Storage and transport without original packaging	0 °C/32 °F to +35 °C/95 °F max. 70 % relative humidity
Operation	0 °C/32 °F to +35 °C/95 °F max. 100 % relative humidity
Storage	0 °C/32 °F to +35 °C/95 °F max. 70 % relative humidity and protect from direct sunlight.

12. Configuration of the length compensation of Modules


Please see the measuring instructions on the macu4 website under the heading Explorer Modules.

13. Returns

If you believe there is a problem with your Explorer Module, please first read this manual carefully before contacting support@macu4.com or your prosthesis manufacturer (unless you purchased the product directly from Macu). Please use the Product Experience Report (PER): https://25462115.fs1.hubspotusercontent-eu1.net/hubfs/25462115/mac4_AG_Explorer/Product_Experience_Report_EN.pdf

Information on the UDI-DI and UDI-PI of your product. These can be found imprinted on the Module.

Valid for BALL, SWIM & TWIN and all Module with a spacer, and RING

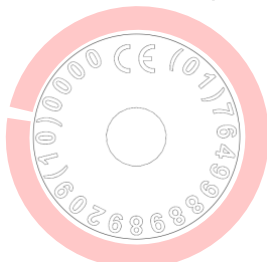
UDI-DI = (01)7649988989209 (SWIM)
UDI-PI = (10) _____

BIKE Standard without spacer

UDI-DI = (01)7649988373206
UDI-PI = (10) _____

PINCH Standard without spacer

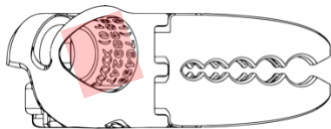UDI-DI = (01)7649988989247
UDI-PI = (10) _____

If you return your Module to us or your prosthetist, please pack it appropriately; the original packaging in which you received your Module is perfectly suitable for this purpose.

14. Disposal of products

If you are no longer going to use a product, you can dispose of it as follows:

- Return to macu4 AG
- See FAQs on the website Website www.macu4.ch

15. Document history

Version	Date	Description
1.0	28.12.2021	Creation
2.0	07.02.2022	Update
3.0	01.05.2022	Update
4.0	23.08.2022	Update Chapter 9 Conformity and Chapter 13 Returns
5.0	30.12.2022	Update Chapter 9 and Chapter 13
6.0	15.04.2023	Update Chapter 9
7.0	01.07.2023	Update Chapters 2-4; 5.2; 11.3
8.0	02.08.2023	Update Chapters 2-4; 10; 12
9.0	01.02.2024	Update Chapters 2;3 ; 9; 11.1; 13, New Chapter 4

macu4™ | Instructions for use

Explorer System – Modules and Ring

Annex I

Explorer RING EXP – Articles with Coupling M12x1.5		
01013000	Explorer RING EXP	Coupling M12x1.5
Explorer RING EXP – Articles with Coupling 1/2"-20		
01014000	Explorer RING EXP	Coupling 1/2"-20
Explorer RING STD – Articles with Coupling M12x1.5		
01071000	Explorer RING STD	Coupling M12x1.5
Explorer RING STD – Articles with Coupling 1/2"-20		
01072000	Explorer RING STD	Coupling 1/2"-20
Explorer BALL Module Standard – Articles with Coupling M12x1.5		
01031000	Explorer BALL Module Standard without spacer	Coupling M12x1.5
01031016	Explorer BALL Module Standard with spacer	16mm Length, Coupling M12x1.5
01031020	Explorer BALL Module Standard with spacer	20mm Length, Coupling M12x1.5
01031025	Explorer BALL Module Standard with spacer	25mm Length, Coupling M12x1.5
01031030	Explorer BALL Module Standard with spacer	30mm Length, Coupling M12x1.5
01031035	Explorer BALL Module Standard with spacer	35mm Length, Coupling M12x1.5
01031040	Explorer BALL Module Standard with spacer	40mm Length, Coupling M12x1.5
01031045	Explorer BALL Module Standard with spacer	45mm Length, Coupling M12x1.5
01031050	Explorer BALL Module Standard with spacer	50mm Length, Coupling M12x1.5
01031055	Explorer BALL Module Standard with spacer	55mm Length, Coupling M12x1.5
01031060	Explorer BALL Module Standard with spacer	60mm Length, Coupling M12x1.5
01031065	Explorer BALL Module Standard with spacer	65mm Length, Coupling M12x1.5
01031070	Explorer BALL Module Standard with spacer	70mm Length, Coupling M12x1.5
01031075	Explorer BALL Module Standard with spacer	75mm Length, Coupling M12x1.5
01031080	Explorer BALL Module Standard with spacer	80mm Length, Coupling M12x1.5
01031085	Explorer BALL Module Standard with spacer	85mm Length, Coupling M12x1.5
01031090	Explorer BALL Module Standard with spacer	90mm Length, Coupling M12x1.5
01031095	Explorer BALL Module Standard with spacer	95mm Length, Coupling M12x1.5
01031100	Explorer BALL Module Standard with spacer	100mm Length, Coupling M12x1.5
01031105	Explorer BALL Module Standard with spacer	105mm Length, Coupling M12x1.5
01031110	Explorer BALL Module Standard with spacer	110mm Length, Coupling M12x1.5
01031115	Explorer BALL Module Standard with spacer	115mm Length, Coupling M12x1.5
01031120	Explorer BALL Module Standard with spacer	120mm Length, Coupling M12x1.5
01031125	Explorer BALL Module Standard with spacer	125mm Length, Coupling M12x1.5
01031130	Explorer BALL Module Standard with spacer	130mm Length, Coupling M12x1.5
01031135	Explorer BALL Module Standard with spacer	135mm Length, Coupling M12x1.5
01031140	Explorer BALL Module Standard with spacer	140mm Length, Coupling M12x1.5

macu4™ | Instructions for use

Explorer System – Modules and Ring

01031145	Explorer BALL Module Standard with spacer	145mm Length, Coupling M12x1.5
01031150	Explorer BALL Module Standard with spacer	150mm Length, Coupling M12x1.5
01031155	Explorer BALL Module Standard with spacer	155mm Length, Coupling M12x1.5
01031160	Explorer BALL Module Standard with spacer	160mm Length, Coupling M12x1.5
01031165	Explorer BALL Module Standard with spacer	165mm Length, Coupling M12x1.5
01031170	Explorer BALL Module Standard with spacer	170mm Length, Coupling M12x1.5
01031175	Explorer BALL Module Standard with spacer	175mm Length, Coupling M12x1.5
01031180	Explorer BALL Module Standard with spacer	180mm Length, Coupling M12x1.5
01031185	Explorer BALL Module Standard with spacer	185mm Length, Coupling M12x1.5
01031190	Explorer BALL Module Standard with spacer	190mm Length, Coupling M12x1.5
01031195	Explorer BALL Module Standard with spacer	195mm Length, Coupling M12x1.5

Explorer BALL Module Standard – Articles with Coupling 1/2"-20		
01032000	Explorer BALL Module Standard without Spacer	Coupling 1/2"-20
01032016	Explorer BALL Module Standard with spacer	16mm Length, Coupling 1/2"-20
01032020	Explorer BALL Module Standard with spacer	20mm Length, Coupling 1/2"-20
01032025	Explorer BALL Module Standard with spacer	25mm Length, Coupling 1/2"-20
01032030	Explorer BALL Module Standard with spacer	30mm Length, Coupling 1/2"-20
01032035	Explorer BALL Module Standard with spacer	35mm Length, Coupling 1/2"-20
01032040	Explorer BALL Module Standard with spacer	40mm Length, Coupling 1/2"-20
01032045	Explorer BALL Module Standard with spacer	45mm Length, Coupling 1/2"-20
01032050	Explorer BALL Module Standard with spacer	50mm Length, Coupling 1/2"-20
01032055	Explorer BALL Module Standard with spacer	55mm Length, Coupling 1/2"-20
01032060	Explorer BALL Module Standard with spacer	60mm Length, Coupling 1/2"-20
01032065	Explorer BALL Module Standard with spacer	65mm Length, Coupling 1/2"-20
01032070	Explorer BALL Module Standard with spacer	70mm Length, Coupling 1/2"-20
01032075	Explorer BALL Module Standard with spacer	75mm Length, Coupling 1/2"-20
01032080	Explorer BALL Module Standard with spacer	80mm Length, Coupling 1/2"-20
01032085	Explorer BALL Module Standard with spacer	85mm Length, Coupling 1/2"-20
01032090	Explorer BALL Module Standard with spacer	90mm Length, Coupling 1/2"-20
01032095	Explorer BALL Module Standard with spacer	95mm Length, Coupling 1/2"-20
01032100	Explorer BALL Module Standard with spacer	100mm Length, Coupling 1/2"-20
01032105	Explorer BALL Module Standard with spacer	105mm Length, Coupling 1/2"-20
01032110	Explorer BALL Module Standard with spacer	110mm Length, Coupling 1/2"-20
01032115	Explorer BALL Module Standard with spacer	115mm Length, Coupling 1/2"-20
01032120	Explorer BALL Module Standard with spacer	120mm Length, Coupling 1/2"-20
01032125	Explorer BALL Module Standard with spacer	125mm Length, Coupling 1/2"-20
01032130	Explorer BALL Module Standard with spacer	130mm Length, Coupling 1/2"-20
01032135	Explorer BALL Module Standard with spacer	135mm Length, Coupling 1/2"-20
01032140	Explorer BALL Module Standard with spacer	140mm Length, Coupling 1/2"-20

macu4™ | Instructions for use

Explorer System – Modules and Ring

01032145	Explorer BALL Module Standard with spacer	145mm Length, Coupling 1/2"-20
01032150	Explorer BALL Module Standard with spacer	150mm Length, Coupling 1/2"-20
01032155	Explorer BALL Module Standard with spacer	155mm Length, Coupling 1/2"-20
01032160	Explorer BALL Module Standard with spacer	160mm Length, Coupling 1/2"-20
01032165	Explorer BALL Module Standard with spacer	165mm Length, Coupling 1/2"-20
01032170	Explorer BALL Module Standard with spacer	170mm Length, Coupling 1/2"-20
01032175	Explorer BALL Module Standard with spacer	175mm Length, Coupling 1/2"-20
01032180	Explorer BALL Module Standard with spacer	180mm Length, Coupling 1/2"-20
01032185	Explorer BALL Module Standard with spacer	185mm Length, Coupling 1/2"-20
01032190	Explorer BALL Module Standard with spacer	190mm Length, Coupling 1/2"-20
01032195	Explorer BALL Module Standard with spacer	195mm Length, Coupling 1/2"-20

Explorer BALL Module Small – Articles with Coupling M12x1.5		
01033000	Explorer BALL Module Small without Spacer	Coupling M12x1.5
01033016	Explorer BALL Module Small with spacer	16mm Length, Coupling M12x1.5
01033020	Explorer BALL Module Small with spacer	20mm Length, Coupling M12x1.5
01033025	Explorer BALL Module Small with spacer	25mm Length, Coupling M12x1.5
01033030	Explorer BALL Module Small with spacer	30mm Length, Coupling M12x1.5
01033035	Explorer BALL Module Small with spacer	35mm Length, Coupling M12x1.5
01033040	Explorer BALL Module Small with spacer	40mm Length, Coupling M12x1.5
01033045	Explorer BALL Module Small with spacer	45mm Length, Coupling M12x1.5
01033050	Explorer BALL Module Small with spacer	50mm Length, Coupling M12x1.5
01033055	Explorer BALL Module Small with spacer	55mm Length, Coupling M12x1.5
01033060	Explorer BALL Module Small with spacer	60mm Length, Coupling M12x1.5
01033065	Explorer BALL Module Small with spacer	65mm Length, Coupling M12x1.5
01033070	Explorer BALL Module Small with spacer	70mm Length, Coupling M12x1.5
01033075	Explorer BALL Module Small with spacer	75mm Length, Coupling M12x1.5
01033080	Explorer BALL Module Small with spacer	80mm Length, Coupling M12x1.5
01033085	Explorer BALL Module Small with spacer	85mm Length, Coupling M12x1.5
01033090	Explorer BALL Module Small with spacer	90mm Length, Coupling M12x1.5
01033095	Explorer BALL Module Small with spacer	95mm Length, Coupling M12x1.5
01033100	Explorer BALL Module Small with spacer	100mm Length, Coupling M12x1.5
01033105	Explorer BALL Module Small with spacer	105mm Length, Coupling M12x1.5
01033110	Explorer BALL Module Small with spacer	110mm Length, Coupling M12x1.5
01033115	Explorer BALL Module Small with spacer	115mm Length, Coupling M12x1.5
01033120	Explorer BALL Module Small with spacer	120mm Length, Coupling M12x1.5
01033125	Explorer BALL Module Small with spacer	125mm Length, Coupling M12x1.5
01033130	Explorer BALL Module Small with spacer	130mm Length, Coupling M12x1.5
01033135	Explorer BALL Module Small with spacer	135mm Length, Coupling M12x1.5
01033140	Explorer BALL Module Small with spacer	140mm Length, Coupling M12x1.5

macu4™ | Instructions for use

Explorer System – Modules and Ring

01033145	Explorer BALL Module Small with spacer	145mm Length, Coupling M12x1.5
01033150	Explorer BALL Module Small with spacer	150mm Length, Coupling M12x1.5
01033155	Explorer BALL Module Small with spacer	155mm Length, Coupling M12x1.5
01033160	Explorer BALL Module Small with spacer	160mm Length, Coupling M12x1.5
01033165	Explorer BALL Module Small with spacer	165mm Length, Coupling M12x1.5
01033170	Explorer BALL Module Small with spacer	170mm Length, Coupling M12x1.5
01033175	Explorer BALL Module Small with spacer	175mm Length, Coupling M12x1.5
01033180	Explorer BALL Module Small with spacer	180mm Length, Coupling M12x1.5
01033185	Explorer BALL Module Small with spacer	185mm Length, Coupling M12x1.5
01033190	Explorer BALL Module Small with spacer	190mm Length, Coupling M12x1.5
01033195	Explorer BALL Module Small with spacer	195mm Length, Coupling M12x1.5

Explorer BALL Module Small – Articles with Coupling 1/2"-20		
01034000	Explorer BALL Module Small without Spacer	Coupling 1/2"-20
01034016	Explorer BALL Module Small with spacer	16mm Length, Coupling 1/2"-20
01034020	Explorer BALL Module Small with spacer	20mm Length, Coupling 1/2"-20
01034025	Explorer BALL Module Small with spacer	25mm Length, Coupling 1/2"-20
01034030	Explorer BALL Module Small with spacer	30mm Length, Coupling 1/2"-20
01034035	Explorer BALL Module Small with spacer	35mm Length, Coupling 1/2"-20
01034040	Explorer BALL Module Small with spacer	40mm Length, Coupling 1/2"-20
01034045	Explorer BALL Module Small with spacer	45mm Length, Coupling 1/2"-20
01034050	Explorer BALL Module Small with spacer	50mm Length, Coupling 1/2"-20
01034055	Explorer BALL Module Small with spacer	55mm Length, Coupling 1/2"-20
01034060	Explorer BALL Module Small with spacer	60mm Length, Coupling 1/2"-20
01034065	Explorer BALL Module Small with spacer	65mm Length, Coupling 1/2"-20
01034070	Explorer BALL Module Small with spacer	70mm Length, Coupling 1/2"-20
01034075	Explorer BALL Module Small with spacer	75mm Length, Coupling 1/2"-20
01034080	Explorer BALL Module Small with spacer	80mm Length, Coupling 1/2"-20
01034085	Explorer BALL Module Small with spacer	85mm Length, Coupling 1/2"-20
01034090	Explorer BALL Module Small with spacer	90mm Length, Coupling 1/2"-20
01034095	Explorer BALL Module Small with spacer	95mm Length, Coupling 1/2"-20
01034100	Explorer BALL Module Small with spacer	100mm Length, Coupling 1/2"-20
01034105	Explorer BALL Module Small with spacer	105mm Length, Coupling 1/2"-20
01034110	Explorer BALL Module Small with spacer	110mm Length, Coupling 1/2"-20
01034115	Explorer BALL Module Small with spacer	115mm Length, Coupling 1/2"-20
01034120	Explorer BALL Module Small with spacer	120mm Length, Coupling 1/2"-20
01034125	Explorer BALL Module Small with spacer	125mm Length, Coupling 1/2"-20
01034130	Explorer BALL Module Small with spacer	130mm Length, Coupling 1/2"-20
01034135	Explorer BALL Module Small with spacer	135mm Length, Coupling 1/2"-20
01034140	Explorer BALL Module Small with spacer	140mm Length, Coupling 1/2"-20

macu4™ | Instructions for use

Explorer System – Modules and Ring

01034145	Explorer BALL Module Small with spacer	145mm Length, Coupling 1/2"-20
01034150	Explorer BALL Module Small with spacer	150mm Length, Coupling 1/2"-20
01034155	Explorer BALL Module Small with spacer	155mm Length, Coupling 1/2"-20
01034160	Explorer BALL Module Small with spacer	160mm Length, Coupling 1/2"-20
01034165	Explorer BALL Module Small with spacer	165mm Length, Coupling 1/2"-20
01034170	Explorer BALL Module Small with spacer	170mm Length, Coupling 1/2"-20
01034175	Explorer BALL Module Small with spacer	175mm Length, Coupling 1/2"-20
01034180	Explorer BALL Module Small with spacer	180mm Length, Coupling 1/2"-20
01034185	Explorer BALL Module Small with spacer	185mm Length, Coupling 1/2"-20
01034190	Explorer BALL Module Small with spacer	190mm Length, Coupling 1/2"-20
01034195	Explorer BALL Module Small with spacer	195mm Length, Coupling 1/2"-20

Explorer BIKE Module Standard – Articles with Coupling M12x1.5		
01021000	Explorer BIKE Module Standard without Spacer	Coupling M12x1.5
01021016	Explorer BIKE Module Standard with Spacer	16mm Length, Coupling M12x1.5
01021020	Explorer BIKE Module Standard with Spacer	20mm Length, Coupling M12x1.5
01021025	Explorer BIKE Module Standard with Spacer	25mm Length, Coupling M12x1.5
01021030	Explorer BIKE Module Standard with Spacer	30mm Length, Coupling M12x1.5
01021035	Explorer BIKE Module Standard with Spacer	35mm Length, Coupling M12x1.5
01021040	Explorer BIKE Module Standard with Spacer	40mm Length, Coupling M12x1.5
01021045	Explorer BIKE Module Standard with Spacer	45mm Length, Coupling M12x1.5
01021050	Explorer BIKE Module Standard with Spacer	50mm Length, Coupling M12x1.5
01021055	Explorer BIKE Module Standard with Spacer	55mm Length, Coupling M12x1.5
01021060	Explorer BIKE Module Standard with Spacer	60mm Length, Coupling M12x1.5
01021065	Explorer BIKE Module Standard with Spacer	65mm Length, Coupling M12x1.5
01021070	Explorer BIKE Module Standard with Spacer	70mm Length, Coupling M12x1.5
01021075	Explorer BIKE Module Standard with Spacer	75mm Length, Coupling M12x1.5
01021080	Explorer BIKE Module Standard with Spacer	80mm Length, Coupling M12x1.5
01021085	Explorer BIKE Module Standard with Spacer	85mm Length, Coupling M12x1.5
01021090	Explorer BIKE Module Standard with Spacer	90mm Length, Coupling M12x1.5
01021095	Explorer BIKE Module Standard with Spacer	95mm Length, Coupling M12x1.5
01021100	Explorer BIKE Module Standard with Spacer	100mm Length, Coupling M12x1.5
01021105	Explorer BIKE Module Standard with Spacer	105mm Length, Coupling M12x1.5
01021110	Explorer BIKE Module Standard with Spacer	110mm Length, Coupling M12x1.5
01021115	Explorer BIKE Module Standard with Spacer	115mm Length, Coupling M12x1.5
01021120	Explorer BIKE Module Standard with Spacer	120mm Length, Coupling M12x1.5
01021125	Explorer BIKE Module Standard with Spacer	125mm Length, Coupling M12x1.5
01021130	Explorer BIKE Module Standard with Spacer	130mm Length, Coupling M12x1.5
01021135	Explorer BIKE Module Standard with Spacer	135mm Length, Coupling M12x1.5
01021140	Explorer BIKE Module Standard with Spacer	140mm Length, Coupling M12x1.5

macu4™ | Instructions for use

Explorer System – Modules and Ring

01021145	Explorer BIKE Module Standard with Spacer	145mm Length, Coupling M12x1.5
01021150	Explorer BIKE Module Standard with Spacer	150mm Length, Coupling M12x1.5
01021155	Explorer BIKE Module Standard with Spacer	155mm Length, Coupling M12x1.5
01021160	Explorer BIKE Module Standard with Spacer	160mm Length, Coupling M12x1.5
01021165	Explorer BIKE Module Standard with Spacer	165mm Length, Coupling M12x1.5
01021170	Explorer BIKE Module Standard with Spacer	170mm Length, Coupling M12x1.5
01021175	Explorer BIKE Module Standard with Spacer	175mm Length, Coupling M12x1.5
01021180	Explorer BIKE Module Standard with Spacer	180mm Length, Coupling M12x1.5
01021185	Explorer BIKE Module Standard with Spacer	185mm Length, Coupling M12x1.5
01021190	Explorer BIKE Module Standard with Spacer	190mm Length, Coupling M12x1.5
01021195	Explorer BIKE Module Standard with Spacer	195mm Length, Coupling M12x1.5

Explorer BIKE Module Standard – Articles with Coupling 1/2"-20		
01022000	Explorer BIKE Module Standard without Spacer	Coupling 1/2"-20
01022016	Explorer BIKE Module Standard with Spacer	16mm Length, Coupling 1/2"-20
01022020	Explorer BIKE Module Standard with Spacer	20mm Length, Coupling 1/2"-20
01022025	Explorer BIKE Module Standard with Spacer	25mm Length, Coupling 1/2"-20
01022030	Explorer BIKE Module Standard with Spacer	30mm Length, Coupling 1/2"-20
01022035	Explorer BIKE Module Standard with Spacer	35mm Length, Coupling 1/2"-20
01022040	Explorer BIKE Module Standard with Spacer	40mm Length, Coupling 1/2"-20
01022045	Explorer BIKE Module Standard with Spacer	45mm Length, Coupling 1/2"-20
01022050	Explorer BIKE Module Standard with Spacer	50mm Length, Coupling 1/2"-20
01022055	Explorer BIKE Module Standard with Spacer	55mm Length, Coupling 1/2"-20
01022060	Explorer BIKE Module Standard with Spacer	60mm Length, Coupling 1/2"-20
01022065	Explorer BIKE Module Standard with Spacer	65mm Length, Coupling 1/2"-20
01022070	Explorer BIKE Module Standard with Spacer	70mm Length, Coupling 1/2"-20
01022075	Explorer BIKE Module Standard with Spacer	75mm Length, Coupling 1/2"-20
01022080	Explorer BIKE Module Standard with Spacer	80mm Length, Coupling 1/2"-20
01022085	Explorer BIKE Module Standard with Spacer	85mm Length, Coupling 1/2"-20
01022090	Explorer BIKE Module Standard with Spacer	90mm Length, Coupling 1/2"-20
01022095	Explorer BIKE Module Standard with Spacer	95mm Length, Coupling 1/2"-20
01022100	Explorer BIKE Module Standard with Spacer	100mm Length, Coupling 1/2"-20
01022105	Explorer BIKE Module Standard with Spacer	105mm Length, Coupling 1/2"-20
01022110	Explorer BIKE Module Standard with Spacer	110mm Length, Coupling 1/2"-20
01022115	Explorer BIKE Module Standard with Spacer	115mm Length, Coupling 1/2"-20
01022120	Explorer BIKE Module Standard with Spacer	120mm Length, Coupling 1/2"-20
01022125	Explorer BIKE Module Standard with Spacer	125mm Length, Coupling 1/2"-20
01022130	Explorer BIKE Module Standard with Spacer	130mm Length, Coupling 1/2"-20
01022135	Explorer BIKE Module Standard with Spacer	135mm Length, Coupling 1/2"-20
01022140	Explorer BIKE Module Standard with Spacer	140mm Length, Coupling 1/2"-20

macu4™ | Instructions for use

Explorer System – Modules and Ring

01022145	Explorer BIKE Module Standard with Spacer	145mm Length, Coupling 1/2"-20
01022150	Explorer BIKE Module Standard with Spacer	150mm Length, Coupling 1/2"-20
01022155	Explorer BIKE Module Standard with Spacer	155mm Length, Coupling 1/2"-20
01022160	Explorer BIKE Module Standard with Spacer	160mm Length, Coupling 1/2"-20
01022165	Explorer BIKE Module Standard with Spacer	165mm Length, Coupling 1/2"-20
01022170	Explorer BIKE Module Standard with Spacer	170mm Length, Coupling 1/2"-20
01022175	Explorer BIKE Module Standard with Spacer	175mm Length, Coupling 1/2"-20
01022180	Explorer BIKE Module Standard with Spacer	180mm Length, Coupling 1/2"-20
01022185	Explorer BIKE Module Standard with Spacer	185mm Length, Coupling 1/2"-20
01022190	Explorer BIKE Module Standard with Spacer	190mm Length, Coupling 1/2"-20
01022195	Explorer BIKE Module Standard with Spacer	195mm Length, Coupling 1/2"-20

Explorer PINCH Module Standard – Articles with Coupling M12x1.5		
01081000	Explorer PINCH Module Standard without Spacer	Coupling M12x1.5
01081016	Explorer PINCH Module Standard with Spacer	16mm Length, Coupling M12x1.5
01081020	Explorer PINCH Module Standard with Spacer	20mm Length, Coupling M12x1.5
01081025	Explorer PINCH Module Standard with Spacer	25mm Length, Coupling M12x1.5
01081030	Explorer PINCH Module Standard with Spacer	30mm Length, Coupling M12x1.5
01081035	Explorer PINCH Module Standard with Spacer	35mm Length, Coupling M12x1.5
01081040	Explorer PINCH Module Standard with Spacer	40mm Length, Coupling M12x1.5
01081045	Explorer PINCH Module Standard with Spacer	45mm Length, Coupling M12x1.5
01081050	Explorer PINCH Module Standard with Spacer	50mm Length, Coupling M12x1.5
01081055	Explorer PINCH Module Standard with Spacer	55mm Length, Coupling M12x1.5
01081060	Explorer PINCH Module Standard with Spacer	60mm Length, Coupling M12x1.5
01081065	Explorer PINCH Module Standard with Spacer	65mm Length, Coupling M12x1.5
01081070	Explorer PINCH Module Standard with Spacer	70mm Length, Coupling M12x1.5
01081075	Explorer PINCH Module Standard with Spacer	75mm Length, Coupling M12x1.5
01081080	Explorer PINCH Module Standard with Spacer	80mm Length, Coupling M12x1.5
01081085	Explorer PINCH Module Standard with Spacer	85mm Length, Coupling M12x1.5
01081090	Explorer PINCH Module Standard with Spacer	90mm Length, Coupling M12x1.5
01081095	Explorer PINCH Module Standard with Spacer	95mm Length, Coupling M12x1.5
01081100	Explorer PINCH Module Standard with Spacer	100mm Length, Coupling M12x1.5
01081105	Explorer PINCH Module Standard with Spacer	105mm Length, Coupling M12x1.5
01081110	Explorer PINCH Module Standard with Spacer	110mm Length, Coupling M12x1.5
01081115	Explorer PINCH Module Standard with Spacer	115mm Length, Coupling M12x1.5
01081120	Explorer PINCH Module Standard with Spacer	120mm Length, Coupling M12x1.5
01081125	Explorer PINCH Module Standard with Spacer	125mm Length, Coupling M12x1.5
01081130	Explorer PINCH Module Standard with Spacer	130mm Length, Coupling M12x1.5
01081135	Explorer PINCH Module Standard with Spacer	135mm Length, Coupling M12x1.5
01081140	Explorer PINCH Module Standard with Spacer	140mm Length, Coupling M12x1.5

macu4™ | Instructions for use

Explorer System – Modules and Ring

01081145	Explorer PINCH Module Standard with Spacer	145mm Length, Coupling M12x1.5
01081150	Explorer PINCH Module Standard with Spacer	150mm Length, Coupling M12x1.5
01081155	Explorer PINCH Module Standard with Spacer	155mm Length, Coupling M12x1.5
01081160	Explorer PINCH Module Standard with Spacer	160mm Length, Coupling M12x1.5
01081165	Explorer PINCH Module Standard with Spacer	165mm Length, Coupling M12x1.5
01081170	Explorer PINCH Module Standard with Spacer	170mm Length, Coupling M12x1.5
01081175	Explorer PINCH Module Standard with Spacer	175mm Length, Coupling M12x1.5
01081180	Explorer PINCH Module Standard with Spacer	180mm Length, Coupling M12x1.5
01081185	Explorer PINCH Module Standard with Spacer	185mm Length, Coupling M12x1.5
01081190	Explorer PINCH Module Standard with Spacer	190mm Length, Coupling M12x1.5
01081195	Explorer PINCH Module Standard with Spacer	195mm Length, Coupling M12x1.5

Explorer PINCH Module Standard – Articles with Coupling 1/2"-20		
01082000	Explorer PINCH Module Standard without Spacer	Coupling 1/2"-20
01082016	Explorer PINCH Module Standard with Spacer	16mm Length, Coupling 1/2"-20
01082020	Explorer PINCH Module Standard with Spacer	20mm Length, Coupling 1/2"-20
01082025	Explorer PINCH Module Standard with Spacer	25mm Length, Coupling 1/2"-20
01082030	Explorer PINCH Module Standard with Spacer	30mm Length, Coupling 1/2"-20
01082035	Explorer PINCH Module Standard with Spacer	35mm Length, Coupling 1/2"-20
01082040	Explorer PINCH Module Standard with Spacer	40mm Length, Coupling 1/2"-20
01082045	Explorer PINCH Module Standard with Spacer	45mm Length, Coupling 1/2"-20
01082050	Explorer PINCH Module Standard with Spacer	50mm Length, Coupling 1/2"-20
01082055	Explorer PINCH Module Standard with Spacer	55mm Length, Coupling 1/2"-20
01082060	Explorer PINCH Module Standard with Spacer	60mm Length, Coupling 1/2"-20
01082065	Explorer PINCH Module Standard with Spacer	65mm Length, Coupling 1/2"-20
01082070	Explorer PINCH Module Standard with Spacer	70mm Length, Coupling 1/2"-20
01082075	Explorer PINCH Module Standard with Spacer	75mm Length, Coupling 1/2"-20
01082080	Explorer PINCH Module Standard with Spacer	80mm Length, Coupling 1/2"-20
01082085	Explorer PINCH Module Standard with Spacer	85mm Length, Coupling 1/2"-20
01082090	Explorer PINCH Module Standard with Spacer	90mm Length, Coupling 1/2"-20
01082095	Explorer PINCH Module Standard with Spacer	95mm Length, Coupling 1/2"-20
01082100	Explorer PINCH Module Standard with Spacer	100mm Length, Coupling 1/2"-20
01082105	Explorer PINCH Module Standard with Spacer	105mm Length, Coupling 1/2"-20
01082110	Explorer PINCH Module Standard with Spacer	110mm Length, Coupling 1/2"-20
01082115	Explorer PINCH Module Standard with Spacer	115mm Length, Coupling 1/2"-20
01082120	Explorer PINCH Module Standard with Spacer	120mm Length, Coupling 1/2"-20
01082125	Explorer PINCH Module Standard with Spacer	125mm Length, Coupling 1/2"-20
01082130	Explorer PINCH Module Standard with Spacer	130mm Length, Coupling 1/2"-20
01082135	Explorer PINCH Module Standard with Spacer	135mm Length, Coupling 1/2"-20
01082140	Explorer PINCH Module Standard with Spacer	140mm Length, Coupling 1/2"-20

macu4™ | Instructions for use

Explorer System – Modules and Ring

01082145	Explorer PINCH Module Standard with Spacer	145mm Length, Coupling 1/2"-20
01082150	Explorer PINCH Module Standard with Spacer	150mm Length, Coupling 1/2"-20
01082155	Explorer PINCH Module Standard with Spacer	155mm Length, Coupling 1/2"-20
01082160	Explorer PINCH Module Standard with Spacer	160mm Length, Coupling 1/2"-20
01082165	Explorer PINCH Module Standard with Spacer	165mm Length, Coupling 1/2"-20
01082170	Explorer PINCH Module Standard with Spacer	170mm Length, Coupling 1/2"-20
01082175	Explorer PINCH Module Standard with Spacer	175mm Length, Coupling 1/2"-20
01082180	Explorer PINCH Module Standard with Spacer	180mm Length, Coupling 1/2"-20
01082185	Explorer PINCH Module Standard with Spacer	185mm Length, Coupling 1/2"-20
01082190	Explorer PINCH Module Standard with Spacer	190mm Length, Coupling 1/2"-20
01082195	Explorer PINCH Module Standard with Spacer	195mm Length, Coupling 1/2"-20

Explorer SWIM Module Standard – Articles with Coupling M12x1.5		
01041000	Explorer SWIM Module Standard without Spacer	Coupling M12x1.5

Explorer SWIM Module Standard – Articles with Coupling 1/2"-20		
01042000	Explorer SWIM Module Standard without Spacer	Coupling 1/2"-20

Explorer SWIM Module Small – Articles with Coupling M12x1.5		
01043000	Explorer SWIM Module Small without Spacer	Coupling M12x1.5

Explorer SWIM Module Small – Articles with Coupling 1/2"-20		
01044000	Explorer SWIM Module Small without Spacer	Coupling 1/2"-20

Explorer TWIN Module Standard – Articles with Coupling M12x1.5		
01051000	Explorer TWIN Module Standard without Spacer	Coupling M12x1.5
01051016	Explorer TWIN Module Standard with Spacer	16mm Length, Coupling M12x1.5
01051020	Explorer TWIN Module Standard with Spacer	20mm Length, Coupling M12x1.5
01051025	Explorer TWIN Module Standard with Spacer	25mm Length, Coupling M12x1.5
01051030	Explorer TWIN Module Standard with Spacer	30mm Length, Coupling M12x1.5
01051035	Explorer TWIN Module Standard with Spacer	35mm Length, Coupling M12x1.5
01051040	Explorer TWIN Module Standard with Spacer	40mm Length, Coupling M12x1.5
01051045	Explorer TWIN Module Standard with Spacer	45mm Length, Coupling M12x1.5
01051050	Explorer TWIN Module Standard with Spacer	50mm Length, Coupling M12x1.5
01051055	Explorer TWIN Module Standard with Spacer	55mm Length, Coupling M12x1.5
01051060	Explorer TWIN Module Standard with Spacer	60mm Length, Coupling M12x1.5
01051065	Explorer TWIN Module Standard with Spacer	65mm Length, Coupling M12x1.5
01051070	Explorer TWIN Module Standard with Spacer	70mm Length, Coupling M12x1.5
01051075	Explorer TWIN Module Standard with Spacer	75mm Length, Coupling M12x1.5
01051080	Explorer TWIN Module Standard with Spacer	80mm Length, Coupling M12x1.5
01051085	Explorer TWIN Module Standard with Spacer	85mm Length, Coupling M12x1.5
01051090	Explorer TWIN Module Standard with Spacer	90mm Length, Coupling M12x1.5

macu4™ | Instructions for use

Explorer System – Modules and Ring

01051095	Explorer TWIN Module Standard with Spacer	95mm Length, Coupling M12x1.5
01051100	Explorer TWIN Module Standard with Spacer	100mm Length, Coupling M12x1.5
01051105	Explorer TWIN Module Standard with Spacer	105mm Length, Coupling M12x1.5
01051110	Explorer TWIN Module Standard with Spacer	110mm Length, Coupling M12x1.5
01051115	Explorer TWIN Module Standard with Spacer	115mm Length, Coupling M12x1.5
01051120	Explorer TWIN Module Standard with Spacer	120mm Length, Coupling M12x1.5
01051125	Explorer TWIN Module Standard with Spacer	125mm Length, Coupling M12x1.5
01051130	Explorer TWIN Module Standard with Spacer	130mm Length, Coupling M12x1.5
01051135	Explorer TWIN Module Standard with Spacer	135mm Length, Coupling M12x1.5
01051140	Explorer TWIN Module Standard with Spacer	140mm Length, Coupling M12x1.5
01051145	Explorer TWIN Module Standard with Spacer	145mm Length, Coupling M12x1.5
01051150	Explorer TWIN Module Standard with Spacer	150mm Length, Coupling M12x1.5
01051155	Explorer TWIN Module Standard with Spacer	155mm Length, Coupling M12x1.5
01051160	Explorer TWIN Module Standard with Spacer	160mm Length, Coupling M12x1.5
01051165	Explorer TWIN Module Standard with Spacer	165mm Length, Coupling M12x1.5
01051170	Explorer TWIN Module Standard with Spacer	170mm Length, Coupling M12x1.5
01051175	Explorer TWIN Module Standard with Spacer	175mm Length, Coupling M12x1.5
01051180	Explorer TWIN Module Standard with Spacer	180mm Length, Coupling M12x1.5
01051185	Explorer TWIN Module Standard with Spacer	185mm Length, Coupling M12x1.5
01051190	Explorer TWIN Module Standard with Spacer	190mm Length, Coupling M12x1.5
01051195	Explorer TWIN Module Standard with Spacer	195mm Length, Coupling M12x1.5

Explorer TWIN Module Standard – Articles with Coupling 1/2"-20		
01052000	Explorer TWIN Module Standard without Spacer	Coupling 1/2"-20
01052016	Explorer TWIN Module Standard with Spacer	16mm Length, Coupling 1/2"-20
01052020	Explorer TWIN Module Standard with Spacer	20mm Length, Coupling 1/2"-20
01052025	Explorer TWIN Module Standard with Spacer	25mm Length, Coupling 1/2"-20
01052030	Explorer TWIN Module Standard with Spacer	30mm Length, Coupling 1/2"-20
01052035	Explorer TWIN Module Standard with Spacer	35mm Length, Coupling 1/2"-20
01052040	Explorer TWIN Module Standard with Spacer	40mm Length, Coupling 1/2"-20
01052045	Explorer TWIN Module Standard with Spacer	45mm Length, Coupling 1/2"-20
01052050	Explorer TWIN Module Standard with Spacer	50mm Length, Coupling 1/2"-20
01052055	Explorer TWIN Module Standard with Spacer	55mm Length, Coupling 1/2"-20
01052060	Explorer TWIN Module Standard with Spacer	60mm Length, Coupling 1/2"-20
01052065	Explorer TWIN Module Standard with Spacer	65mm Length, Coupling 1/2"-20
01052070	Explorer TWIN Module Standard with Spacer	70mm Length, Coupling 1/2"-20
01052075	Explorer TWIN Module Standard with Spacer	75mm Length, Coupling 1/2"-20
01052080	Explorer TWIN Module Standard with Spacer	80mm Length, Coupling 1/2"-20
01052085	Explorer TWIN Module Standard with Spacer	85mm Length, Coupling 1/2"-20
01052090	Explorer TWIN Module Standard with Spacer	90mm Length, Coupling 1/2"-20

macu4™ | Instructions for use

Explorer System – Modules and Ring

01052095	Explorer TWIN Module Standard with Spacer	95mm Length, Coupling 1/2"-20
01052100	Explorer TWIN Module Standard with Spacer	100mm Length, Coupling 1/2"-20
01052105	Explorer TWIN Module Standard with Spacer	105mm Length, Coupling 1/2"-20
01052110	Explorer TWIN Module Standard with Spacer	110mm Length, Coupling 1/2"-20
01052115	Explorer TWIN Module Standard with Spacer	115mm Length, Coupling 1/2"-20
01052120	Explorer TWIN Module Standard with Spacer	120mm Length, Coupling 1/2"-20
01052125	Explorer TWIN Module Standard with Spacer	125mm Length, Coupling 1/2"-20
01052130	Explorer TWIN Module Standard with Spacer	130mm Length, Coupling 1/2"-20
01052135	Explorer TWIN Module Standard with Spacer	135mm Length, Coupling 1/2"-20
01052140	Explorer TWIN Module Standard with Spacer	140mm Length, Coupling 1/2"-20
01052145	Explorer TWIN Module Standard with Spacer	145mm Length, Coupling 1/2"-20
01052150	Explorer TWIN Module Standard with Spacer	150mm Length, Coupling 1/2"-20
01052155	Explorer TWIN Module Standard with Spacer	155mm Length, Coupling 1/2"-20
01052160	Explorer TWIN Module Standard with Spacer	160mm Length, Coupling 1/2"-20
01052165	Explorer TWIN Module Standard with Spacer	165mm Length, Coupling 1/2"-20
01052170	Explorer TWIN Module Standard with Spacer	170mm Length, Coupling 1/2"-20
01052175	Explorer TWIN Module Standard with Spacer	175mm Length, Coupling 1/2"-20
01052180	Explorer TWIN Module Standard with Spacer	180mm Length, Coupling 1/2"-20
01052185	Explorer TWIN Module Standard with Spacer	185mm Length, Coupling 1/2"-20
01052190	Explorer TWIN Module Standard with Spacer	190mm Length, Coupling 1/2"-20
01052195	Explorer TWIN Module Standard with Spacer	195mm Length, Coupling 1/2"-20

Explorer TWIN Module Large – Articles with Coupling M12x1.5		
01053000	Explorer TWIN Module Large without Spacer	Coupling M12x1.5
01053016	Explorer TWIN Module Large with Spacer	16mm Length, Coupling M12x1.5
01053020	Explorer TWIN Module Large with Spacer	20mm Length, Coupling M12x1.5
01053025	Explorer TWIN Module Large with Spacer	25mm Length, Coupling M12x1.5
01053030	Explorer TWIN Module Large with Spacer	30mm Length, Coupling M12x1.5
01053035	Explorer TWIN Module Large with Spacer	35mm Length, Coupling M12x1.5
01053040	Explorer TWIN Module Large with Spacer	40mm Length, Coupling M12x1.5
01053045	Explorer TWIN Module Large with Spacer	45mm Length, Coupling M12x1.5
01053050	Explorer TWIN Module Large with Spacer	50mm Length, Coupling M12x1.5
01053055	Explorer TWIN Module Large with Spacer	55mm Length, Coupling M12x1.5
01053060	Explorer TWIN Module Large with Spacer	60mm Length, Coupling M12x1.5
01053065	Explorer TWIN Module Large with Spacer	65mm Length, Coupling M12x1.5
01053070	Explorer TWIN Module Large with Spacer	70mm Length, Coupling M12x1.5
01053075	Explorer TWIN Module Large with Spacer	75mm Length, Coupling M12x1.5
01053080	Explorer TWIN Module Large with Spacer	80mm Length, Coupling M12x1.5
01053085	Explorer TWIN Module Large with Spacer	85mm Length, Coupling M12x1.5
01053090	Explorer TWIN Module Large with Spacer	90mm Length, Coupling M12x1.5

macu4™ | Instructions for use

Explorer System – Modules and Ring

01053095	Explorer TWIN Module Large with Spacer	95mm Length, Coupling M12x1.5
01053100	Explorer TWIN Module Large with Spacer	100mm Length, Coupling M12x1.5
01053105	Explorer TWIN Module Large with Spacer	105mm Length, Coupling M12x1.5
01053110	Explorer TWIN Module Large with Spacer	110mm Length, Coupling M12x1.5
01053115	Explorer TWIN Module Large with Spacer	115mm Length, Coupling M12x1.5
01053120	Explorer TWIN Module Large with Spacer	120mm Length, Coupling M12x1.5
01053125	Explorer TWIN Module Large with Spacer	125mm Length, Coupling M12x1.5
01053130	Explorer TWIN Module Large with Spacer	130mm Length, Coupling M12x1.5
01053135	Explorer TWIN Module Large with Spacer	135mm Length, Coupling M12x1.5
01053140	Explorer TWIN Module Large with Spacer	140mm Length, Coupling M12x1.5
01053145	Explorer TWIN Module Large with Spacer	145mm Length, Coupling M12x1.5
01053150	Explorer TWIN Module Large with Spacer	150mm Length, Coupling M12x1.5
01053155	Explorer TWIN Module Large with Spacer	155mm Length, Coupling M12x1.5
01053160	Explorer TWIN Module Large with Spacer	160mm Length, Coupling M12x1.5
01053165	Explorer TWIN Module Large with Spacer	165mm Length, Coupling M12x1.5
01053170	Explorer TWIN Module Large with Spacer	170mm Length, Coupling M12x1.5
01053175	Explorer TWIN Module Large with Spacer	175mm Length, Coupling M12x1.5
01053180	Explorer TWIN Module Large with Spacer	180mm Length, Coupling M12x1.5
01053185	Explorer TWIN Module Large with Spacer	185mm Length, Coupling M12x1.5
01053190	Explorer TWIN Module Large with Spacer	190mm Length, Coupling M12x1.5
01053195	Explorer TWIN Module Large with Spacer	195mm Length, Coupling M12x1.5

Explorer TWIN Module Large – Articles with Coupling 1/2"-20		
01054000	Explorer TWIN Module Large without Spacer	Coupling 1/2"-20
01054016	Explorer TWIN Module Large with Spacer	16mm Length, Coupling 1/2"-20
01054020	Explorer TWIN Module Large with Spacer	20mm Length, Coupling 1/2"-20
01054025	Explorer TWIN Module Large with Spacer	25mm Length, Coupling 1/2"-20
01054030	Explorer TWIN Module Large with Spacer	30mm Length, Coupling 1/2"-20
01054035	Explorer TWIN Module Large with Spacer	35mm Length, Coupling 1/2"-20
01054040	Explorer TWIN Module Large with Spacer	40mm Length, Coupling 1/2"-20
01054045	Explorer TWIN Module Large with Spacer	45mm Length, Coupling 1/2"-20
01054050	Explorer TWIN Module Large with Spacer	50mm Length, Coupling 1/2"-20
01054055	Explorer TWIN Module Large with Spacer	55mm Length, Coupling 1/2"-20
01054060	Explorer TWIN Module Large with Spacer	60mm Length, Coupling 1/2"-20
01054065	Explorer TWIN Module Large with Spacer	65mm Length, Coupling 1/2"-20
01054070	Explorer TWIN Module Large with Spacer	70mm Length, Coupling 1/2"-20
01054075	Explorer TWIN Module Large with Spacer	75mm Length, Coupling 1/2"-20
01054080	Explorer TWIN Module Large with Spacer	80mm Length, Coupling 1/2"-20
01054085	Explorer TWIN Module Large with Spacer	85mm Length, Coupling 1/2"-20
01054090	Explorer TWIN Module Large with Spacer	90mm Length, Coupling 1/2"-20

macu4™ | Instructions for use

Explorer System – Modules and Ring

01054095	Explorer TWIN Module Large with Spacer	95mm Length, Coupling 1/2"-20
01054100	Explorer TWIN Module Large with Spacer	100mm Length, Coupling 1/2"-20
01054105	Explorer TWIN Module Large with Spacer	105mm Length, Coupling 1/2"-20
01054110	Explorer TWIN Module Large with Spacer	110mm Length, Coupling 1/2"-20
01054115	Explorer TWIN Module Large with Spacer	115mm Length, Coupling 1/2"-20
01054120	Explorer TWIN Module Large with Spacer	120mm Length, Coupling 1/2"-20
01054125	Explorer TWIN Module Large with Spacer	125mm Length, Coupling 1/2"-20
01054130	Explorer TWIN Module Large with Spacer	130mm Length, Coupling 1/2"-20
01054135	Explorer TWIN Module Large with Spacer	135mm Length, Coupling 1/2"-20
01054140	Explorer TWIN Module Large with Spacer	140mm Length, Coupling 1/2"-20
01054145	Explorer TWIN Module Large with Spacer	145mm Length, Coupling 1/2"-20
01054150	Explorer TWIN Module Large with Spacer	150mm Length, Coupling 1/2"-20
01054155	Explorer TWIN Module Large with Spacer	155mm Length, Coupling 1/2"-20
01054160	Explorer TWIN Module Large with Spacer	160mm Length, Coupling 1/2"-20
01054165	Explorer TWIN Module Large with Spacer	165mm Length, Coupling 1/2"-20
01054170	Explorer TWIN Module Large with Spacer	170mm Length, Coupling 1/2"-20
01054175	Explorer TWIN Module Large with Spacer	175mm Length, Coupling 1/2"-20
01054180	Explorer TWIN Module Large with Spacer	180mm Length, Coupling 1/2"-20
01054185	Explorer TWIN Module Large with Spacer	185mm Length, Coupling 1/2"-20
01054190	Explorer TWIN Module Large with Spacer	190mm Length, Coupling 1/2"-20
01054195	Explorer TWIN Module Large with Spacer	195mm Length, Coupling 1/2"-20