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Instruction for Use | EN

1. Vorwort

Gültig ab: 25.08.2025

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1. Foreword

Valid from: 25.08.2025

- Read this document carefully before using the product.
- Follow carefully the warnings and instructions for assembly to avoid product damage.
- Medical personnel should assemble the product.
- Contact the manufacturer if you have any questions about the product (e.g., commissioning, use, maintenance, unexpected operation, or incidents).
- Keep this document in a safe place.

These instructions for assembly provide you with important information on the use, adjustment, and handling of the provided component(s).

2. Product

Product service description

The Lynk Interface is a semi-finished part which can be integrated into a custom-made orthosis or prosthetic socket. It must be only used together with Lynk Modules.

Our role is limited to supplying the Interface as a non-finished component. The medical personnel is solely responsible for the integration of this component with other materials or parts, the assembly of the complete prosthetic device, and its clinical evaluation and fitting for the end user. In this capacity, the medical personnel acts as the Legal Manufacturer of the final medical device under applicable medical device regulations and assumes full responsibility for regulatory compliance, safety, and performance of the finished device.

CAUTION

3. Warnings

FAILURE TO HEED THESE WARNINGS OR TO FOLLOW THE INSTRUCTIONS FOR ASSEMBLY COULD RESULT IN MALFUNCTION OF THE PRODUCT OR PHYSICAL INJURY. IN THE EVENT OF INJURY, SEEK MEDICAL ATTENTION PROMPTLY.

Notes on unpacking the supplied component

Only open the packaging at the indicated locations with care so that the component inside is not damaged. Inspect the delivered component and be sure that it is undamaged.

Notes on the lamination process

Make sure that the Interface does not deform, for example, when you work with high temperatures or high pressures applied to the Interface.

Notes on using the connector

Make sure that you do not weaken the Interface, for example, when you decide to use cutting or abrasive processes.

Make sure that no material remains inside the Interface before using it together with a Lynk Module.

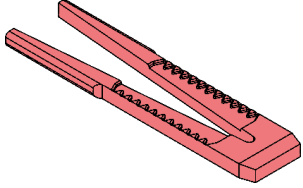
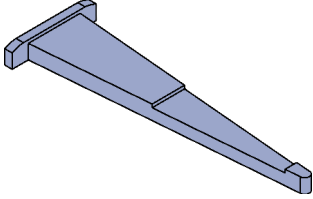
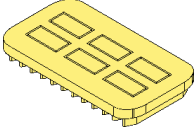
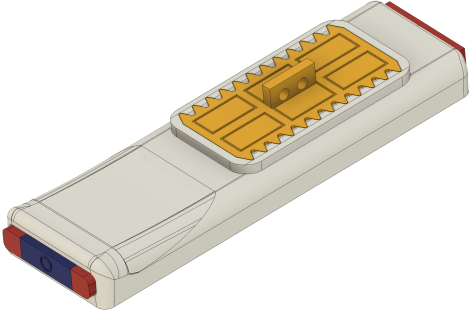
4. Approach

Material needed

Interface	1x assembly material set (content: 1x Interface for lamination, 3x Interface dummies)
Additional materials needed on behalf of medical personnel	Sealing compound / wax

Lynk Interface Dummies

The dummies serve to protect the integrity of the Lynk Interface. There are 3 types of dummies which shall be used.

<p>Part 1.1 – Dummy A Used to be inserted from the distal end</p>	<p>Part 1.2 – Dummy B Used to be inserted from the distal end into the Dummy A</p>	<p>Part 2 –Dummy C Used to be inserted from the top</p>
		
<p>Lynk Interface assembled with Dummies</p>		

Example to illustrate how to use the component

The present steps below illustrate an example of lamination with acrylic resin and unidirectional carbon fibres. The photos are used exemplary.

Preparation of the model

- Create a plaster cast of the limb. The limb shall stand in the desired position.
- Place the first layer of foil around it.
- Create a base layer. Here, a white fabric hose is used. It serves to prevent direct skin contact with the carbon fibres

First application of carbon fibres

- Place the first layer of unidirectional carbon fibre.
- Define the desired orientation of the Lynk Interface. An inserted plate or a Lynk module may help imagine the final result.

macu4™ | Assembly Instructions

Lynk System – Interface for Lamination

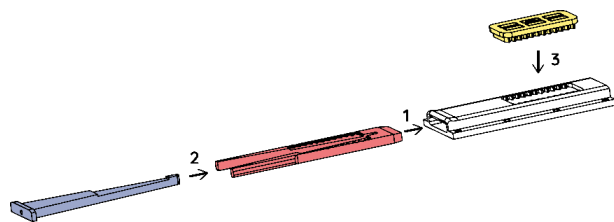
- Use some material such as plaster or silicone paste (Sheraduet) to create a small flat area.
- Consider that the modules have enough space to be inserted and removed from the interface without colliding with the rest of the orthosis.

Warning: Pay attention to the direction of the orientation of the Interface. The proximal side needs to face the proximal side of the forearm model.

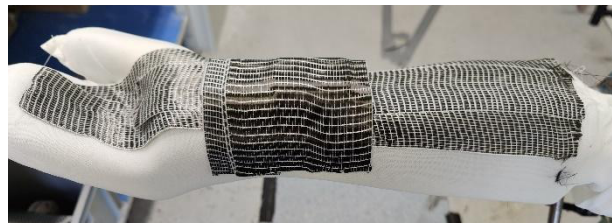
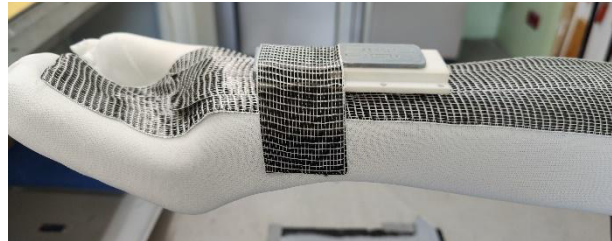
Usage of the Dummies

- Insert the dummies in the Lynk Interface. Follow the indicated order of inserting the dummies. Seal the openings with stick wax.

Second application of carbon fibres

- Apply stepwise the required layers of carbon fibres.

Usage of resin

- Pour resin between the two foils. Here, blue acrylic resin is used.

Hardening of material

- Let it harden the material. For acrylic resin about one hour shall be considered. Please refer to the instructions of the material you are using.

macu4™ | Assembly Instructions

Lynk System – Interface for Lamination

Removal of dummies

- Grind free the distal opening in which Dummy A and B have been inserted and the opening for Dummy C.
- First, remove Dummy C by pulling it out.

- Remove the Dummy B. There is a small hole in the middle. Break the bottom of the hole and screw in the provided M3 screw and pull on it.

- Remove Dummy A. It has a V-shape and this allows you to press the V-edges slightly together. This will help you to remove Dummy A. A sharp or pointy object like a screw drive can be used.

Final cleaning

- Clean the Lynk Interface from residual wax. Cleaning the Lynk Interface is crucial and required to assure compatibility with the Lynk Modules.

Testing

- Confirm the functionality by connecting a Lynk Module.
- If inserting or operating the module is too difficult, further cleaning of the channel of the Lynk Interface is required.

macu4™ | Assembly Instructions

Lynk System – Interface for Lamination

Completion

- Smoothen the grinding locations to prevent sharp edges.
- Apply adapted cushioning and a closing mechanism.

5. Technical data

General	
Drawing (overview)	<p>proximal side</p> <p>distal side</p>
Drawing (front)	
Drawing (side)	
Drawing (top)	
Dimensions (bottom)	
Dimension (side)	
Net weight	Between approx. 30 g / 1.05 oz
Material of printed component	Module body: PA2200, coloured with DM standard colour (DyeMansion)
Lifetime of the 3D printed component	2 years at an average of 120 minutes per day of daily use on the 3D printed component

6. Legal notice

Limitation or exclusion of liability

Failure to follow these instructions for use, care, maintenance, handling, storage, etc., of this product, or the use of this product in any component combination other than those approved by macu4 AG (hereinafter referred to as “Macu”) constitutes misuse of the product and invalidates any and all warranties, express or implied, in their entirety as to any consequences of such misuse. In the event of such misuse, Macu disclaims liability for any adverse consequences to the maximum extent allowed by law. Any opening, disassembly, or repair of the products may only be carried out by Macu or with Macu’s written permission.

Manufacturer of the semi-finished component

Macu4 AG, Rämistrasse 18, 8001 Zurich, Switzerland

Trademark

All designations mentioned in this document are subject without restriction to the provisions of the applicable trademark law and the rights of the respective owners. All trademarks, trade names or company names mentioned herein may be registered trademarks and are subject to the rights of their respective owners. The absence of explicit identification of the trademarks used in this document does not imply that a designation is free of third-party rights.

7. Environmental conditions

Conditions	
During transport	<ul style="list-style-type: none">• Protect from direct sunlight (UV radiation)• Temperature shall be 0 °C to +35 °C / 32 °F to 95 °F

8. Allergic reactions

THIS PRODUCT IS FORMULATED FOR GENERAL SKIN COMPATIBILITY. HOWEVER, ALLERGIC REACTIONS MAY OCCUR IN SOME INDIVIDUALS, PARTICULARLY THOSE WITH A HISTORY OF SKIN ALLERGIES OR SENSITIVITY. THE PRODUCT COMPONENTS MUST NOT BE USED IF THE INDIVIDUAL HAS A KNOWN ALLERGY OR HAS PREVIOUSLY EXPERIENCED A REACTION TO ANY OF THE MATERIALS USED IN THIS PRODUCT. DISCONTINUE USE IMMEDIATELY AND SEEK MEDICAL ADVICE IF REDNESS, ITCHING, RASH, SWELLING, OR IRRITATION OCCURS. INDIVIDUALS WITH SENSITIVE SKIN SHOULD REVIEW THE FULL LIST OF MATERIALS AND, IF NECESSARY, PERFORM A PATCH TEST BEFORE FIRST USE. FOR DETAILED INFORMATION ON THE MATERIALS USED IN SPECIFIC PRODUCTS, PLEASE CONTACT MACU. THE TECHNICAL DATA SECTION PROVIDES GENERAL INFORMATION ON MATERIAL USAGE ONLY.

9. Returns

If you suspect or believe there is a problem with the semi-finished component, please first read this manual carefully before contacting support@macu4.com.. Please use the Product Experience Report (PER):

https://25462115.fs1.hubspotusercontent-eu1.net/hubfs/25462115/mac4_AG_Lynk/Product_Experience_Report_EN.pdf