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Instruction for Use | EN

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Bruksanvisning | NO

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1. Vorwort

Gültig ab: 13.06.2025

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1. Foreword

Valid from: 13.06.2025

- Read this document carefully before using the product.
- Observe the safety instructions to avoid injuries and product damage.
- Instruct the user on the proper and safe use of the product.
- Contact the manufacturer if you have any questions about the product (e.g., commissioning, use, maintenance, unexpected operation or incidents). You will find the contact details on the back.
- Keep this document in a safe place.

These instructions for use provide you with important information on the use, adjustment and handling of the Lumo Ortho-prosthetic System. **Please notice that in addition to these Instructions for use, there are several explanation videos available. See <https://macu4.com/en/application-lumo>.**

2. Use

The Lumo System offers two possibilities for the medical device (here: Base) that is attached to the limb, consisting of either a cuff design or a socket design. Both offer the possibility to couple accessories (here: Modules). Modules are connected and disconnected from the Base. The Base and Modules support the user passively and for specific functions determined by the Modules in combination with the Base. Various exchangeable Modules are available. When used together, they support low-demanding activities.

Intended Use


The Lumo System is intended for users with partial or complete loss of hand function due to temporary or permanent weakened hand or hand injury, paralysis, partial hand absence, wrist disarticulation absence or partial forearm and hand absence. By using the Base with a Module, a lack of gripping dexterity, capability or force can be replaced or supported.

Indications

- The system can be used for unilateral amputees from amputation height below the elbow (according to ISO 8549-2:2020).
- The system can be used for unilateral amputees from amputation height below the wrist (according to ISO 8549-2:2020).
- The system can be used for people with unilateral congenital hand or forearm deficiencies (according to ISO 8549-2:2020).
- The system is suitable for persons with impaired or missing gripping function.
- The system is suitable for persons 12 months old and older. An individual evaluation by the specialist is recommended to assess whether the user is physically and mentally in the condition to use the system properly.

Contraindications

- Not indicated for kids under 12 months old. Exceptions: The professional (e.g. doctor, therapist, clinical orthotist prosthetist) can assess whether the user is able to use the system anyway.
- Not indicated for persons with bilateral forearm or hand deficits. Exceptions: The specialist (e.g. doctor, therapist, clinical orthotist prosthetist) can assess whether the user is able to use the system partially despite this.
- Not for persons with cognitive and/or psychological conditions that would not allow for, or only partially allow for, the indicated use of the Lumo system (e.g. dementia, Alzheimer's, psychoses). Exceptions: The specialist (e.g. doctor, therapist, clinical orthotist prosthetist) can assess whether the user is able to use the system partially despite this.
- Not for persons for whom the Lumo system is expected to place too much mechanical stress on adjacent joints due to the indicated user requirements.
- Not for persons with neurological diseases in which learning movements using the Lumo system would be impaired.
- Not for individuals with fresh wounds, allergies or sensitive skin on the arm.
- Not for individuals with a UBWD combined with a forearm circumference below 110 or above 180 mm, or a wrist circumference below 100 mm or above 135 mm, or a forearm length of less than 120 mm.
- Not for individuals with a UBED combined with a forearm (residual arm) circumference below 140 or above 180 mm, or an arm length below 50 mm.

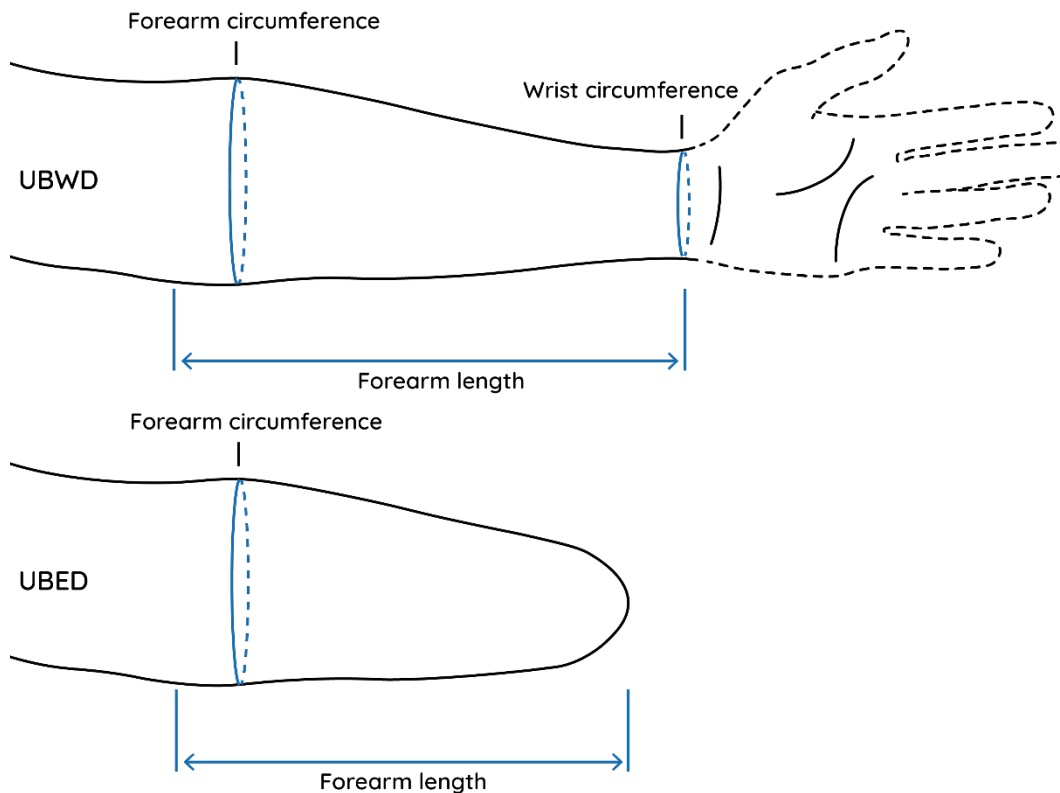Anthropometric values definition.

The system may only be used by users who are physically and mentally fit to operate the system properly. If you are unsure whether the physical and/or mental fitness is sufficient, we recommend an individual assessment by a specialist (e.g. doctor, therapist, prosthetist).

Combination possibilities

The system has been tested for combination possibilities of the Lumo Cuff and Lumo Socket with the different Lynk Clic Heads.

Operating conditions - General

The Lumo System was developed especially for everyday, sports and leisure activities with low to moderate stress. The system must not be used for extraordinary activities. These extraordinary activities include, for example, sports with excessive coupling and/or impact loads or extreme sports. In addition, the system should not be used for driving motor vehicles, driving heavy equipment (e.g., construction machinery), operating industrial machinery or operating motor-driven work equipment.

3. Warnings

General

Carefully review the products before using them to make sure that no area of the product is not damaged. Please always contact support@macu4.com and don't use the component(s) when you have the feeling that, for instance

- a product is damaged.
- a product may not be suitable for an indicated activity.
- a module cannot be coupled easily to the Lumo Cuff or Lumo Socket.
- the Velcro closure is not working properly or releases unintentionally.

Make sure to watch the instructional video for your intended activity. Incorrect use may cause inconvenience or an accident. If you don't find your activity among the instructional videos provided, please contact support@macu4.com or the professional who sold you with the System for instructions before starting the activity.

Use under supervision

The System is suitable for use in children aged 12 months and older. Because children of this age are not able to use the product independently, the System must always be used under the direct supervision of a parent, legal guardian, or responsible adult caregiver. Adult supervision is required at all times during use to ensure safety and proper operation.

Notes on Modules - General

Make sure that you do not overload the Modules. Make sure that you always clean the Modules after use. Remove visible dirt particles so that Modules with flexible modes are not restricted in their function.

Don't use any Module as a means to manage objects that are sharp or that could cause damage in case of loss of control (i.e. razor, drill, etc.).

Don't use a Module if the diameter of the object is too large (see the specifications for each module). If the diameter of an object is too small, you can increase the diameter slightly by using a soft material around the area where you want to attach the module to the object. For example, when using the Hold Module to hold a fork.

Before using a Module, check if it is compatible with the handle (i.e. bike handle, etc.) or object (i.e. fork, etc.), that the Module is tightly secured on the handle or the grip or the object, that you have chosen the correct orientation of the Module on the object/handle.

Notes on Shovel Module

Before using the module, make sure that the screw is tighten and did not get loose. It may get loose in position when the main part of the module is turned frequently. When it gets loose than ensure that the screw is engaging with the nut. Tighten the screw with a 2.5 Allen key.

Notes on Roll Module

Before using the module, make sure that the screw is tighten and did not get loose. It may get loose in position when the main part of the module is turned frequently. When it gets loose than ensure that the screw is engaging with the nut. Tighten the screw with a 2.5 Allen key.

Notes on Hold Module

Before using the module, make sure that the screw of the eccentric cam is tighten and did not get loose. It may get loose in position when the eccentric cam was turned too much counterclockwise. When it gets loose than ensure that the position of the eccentric cam screw is engaging with the nut.

Notes on selection of product size

- Don't use a Cuff or Socket size that is too big for your arm. Any excessive tightening can damage the bendable product parts.
- Don't use a Cuff or Socket size that is too small for your arm. It can lead to painful pressure marks and/or impede vascular flow.

Notes on Cuff and Socket

- Don't use the Cuff or Socket when the protective material is missing.
- Make sure that the Cuff or Socket always sits tight on your forearm. If it gets loose, tighten the Velcro.

Notes on ambient conditions

Make sure that you store or use the products in accordance with the information on temperature, UV radiation and humidity as specified in this instruction. The material properties of the components may change significantly when used and stored at higher or lower temperatures, as stated in this document, and lead to inconvenience or an accident.

Notes on storage conditions


Make sure that the components are well protected during transportation and prolonged storage. Protect especially the Button area of each Module from load situations (e.g. don't place heavy objects on it or apply bending forces). It could deform and/or damage the Module interface. In case of deformation or damage, the Module can no longer be used with the Cuff or Socket.

4. Use of the Components


Before using the Lumo components for the first time, familiarise yourself with the instructions for use. The following chapters explain how to use the Lumo Modules together with the Lumo Cuff or Socket.

4.1. How to use the Lumo Cuff

Design and structure of the Lumo Cuff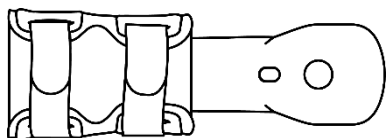


Cuff from the top side equipped with Velcro System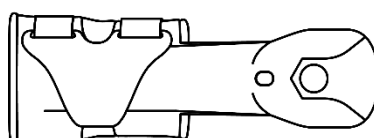

Cuff from the backside (here: with the Hook in its anchoring position)


When using the Cuff for the first time

Make sure that the Cuff sits tight and cannot be moved easily. If the Cuff sits too loose on your arm, you should adapt the Velcro straps. The fit of the Cuff on the arm should be snug but not uncomfortable.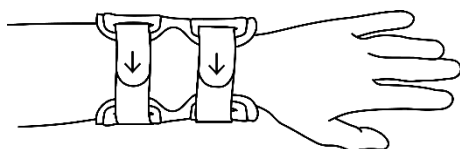


Install Cuff | Open the Velcro strap and pull slightly on the Velcro strap to tighten the sit on your arm.

Put on the Cuff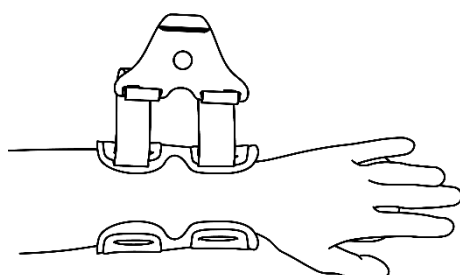

Insert your arm | You can either place the Cuff on a table and insert your arm into it. Or you can use your other hand to hold the Cuff while inserting your arm.

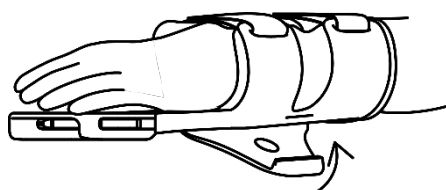Secure the Cuff on your arm | Place the Hook in front of the Hook Anchoring Holes - the magnets will allow it to click into place. Make sure it is well-positioned and secured by gently moving the Cuff.

4.2. How to use the Lumo Socket

When using the Socket for the first time

Make sure that the Socket sits tight and cannot be moved easily. If the Socket sits too loosely on your arm, you should adjust the tension of the Laces. The fit of the Socket on the arm should be snug but not uncomfortable.

Put on the Socket


Insert your arm | You can either place the Socket between your legs and insert your arm into it. Or you can use your other hand to hold the Socket while inserting your arm.

Secure the Socket on your arm | First, pull on the Velcro of the lower shell and, second, on the Velcro of the upper shell. Make sure it is well positioned and secured by gently moving the Socket.

4.3. Final control

- Check the fit of the Cuff or Socket before you start any activity.
- Make sure it is well positioned and secured by trying to move the Cuff or Socket gently. The Cuff or Socket should remain firmly in position.
- If the Cuff or Socket is not tight and can be moved, recheck the Lacing or Velcro System.


4.4. How to remove the Cuff or Socket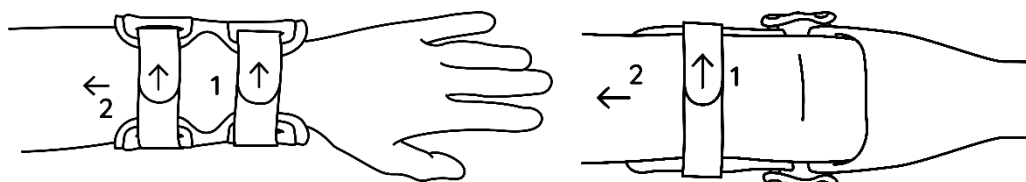

Option for the Cuff | Pull on the Hook. It shall disengage from the Cuff. Once the Hook is free, you can remove the Cuff from your arm.

Option for the Socket | Open the Velcro closure on the upper and lower shells.

4.5. How to use a Module with the Cuff or Socket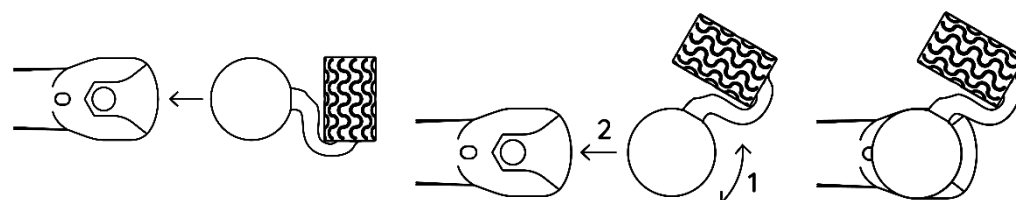

Insert Module | Insert the module into the module receptor. You should hear a small sound.

Change position | Remove the Module and insert it in a different orientation.


Final check | Make sure the Module is well-positioned and secured by trying to move it gently. The Module should remain in position.

5. Lumo Modules


The modules offer no rotation but various locked positions. You can choose between different locked angles.

5.1. Clamp Module

Design and structure of the Module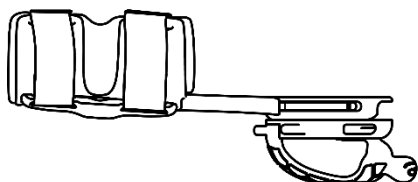

Module coupled to the Cuff.

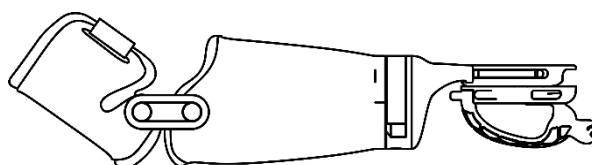
Module coupled to the Socket.

Use case examples with the Module


- **Cutting** | Hold a piece of paper to cut it with scissors.
- **Play** | Hold playing cards.

5.2. Shovel Module

Design and structure of the Module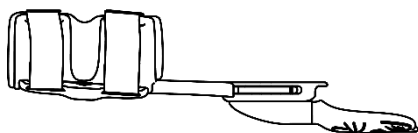

Module coupled to the Cuff.

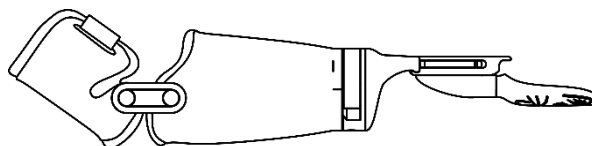Module coupled to the Socket.


Use case examples with the Module

- **Sand castles** | Play in the Sandbox or at the beach.
- **Support** | Hold objects requiring two arms.


Systematically check that the module has no play at the level of the screw. If this is the case, tighten the screw with a 2.5 Allen key.

5.3. Roll Module

Design and structure of the Module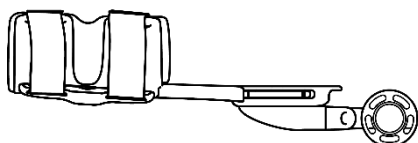

Module coupled to the Cuff.

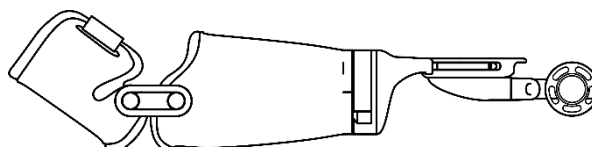Module coupled to the Socket.

Use case examples with the Module


- **Modelling Clay** | Play and spread the clay with the Roll.
- **Music** | The Roll is a maraca (shaker).

Systematically check that the module has no play at the level of the screw. If this is the case, tighten the screw with a 2.5 Allen key.


5.4. Hold Module

Design and structure of the Module


The Module is provided as one single part with a removable module head element.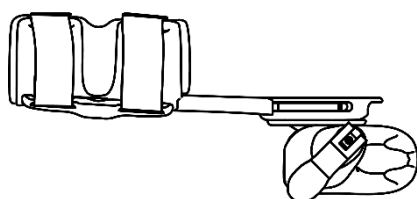

Module coupled to the Cuff.



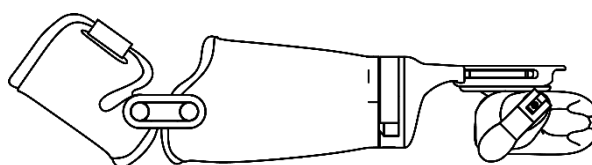
Module coupled to the Socket.

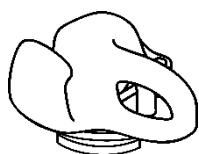Flexible module head | The module head is a clamp-like element and is made from a flexible material to better conform to objects.



Adaptable orientation | You can turn the module head in various directions to find the desired orientation for your activity by releasing the lever.

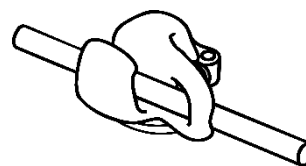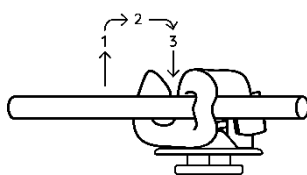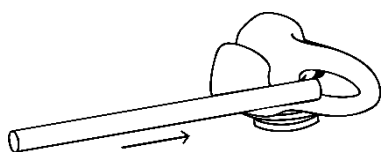Insert the object | Insert the desired object in the hole as indicated by the arrow. Depending on the geometry of the object, it is possible to insert it from the other side.

Securing the object | Follow the steps illustrated by first pulling up the object, second rotating it, and third pressing it into position.

The object is secured | To change the orientation, use the lever.

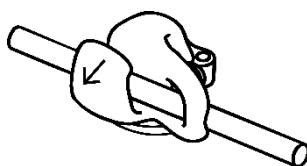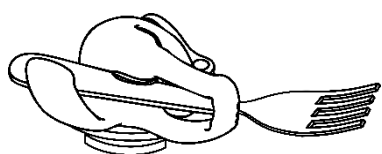Securing the orientation of flat objects | Use the slits to prevent

Pull on the flexible part (here illustrated by the indicator) | The

flat objects from rotating object shall release. You can then involuntarily take it out.

Use case examples with the Module

Always make sure that objects or handles are properly positioned and secured by lifting the object and shaking it gently. Please consult the website of macu4, where you will find specific explanation videos under the section 'Application'.

- **Pens |** For example a pen with a cylindrical handle.
- **Flat objects |** For example a fork with a flat handle.

6. Storage of product

The macu4 product has been specially developed for use with small children. However, it is not a toy and should not be stored or treated as such. To ensure the safety of your child and to maintain the functionality of the product, please observe the following instructions:

- After use, store the product in a clean, dry and safe place – out of the reach of children unless they are under direct supervision.
- Store the product away from areas where it could be mixed with other toys or handled improperly.
- Protect the product from dust, dirt and mechanical impact.
- Avoid storing in places subject to extreme temperature fluctuations or high humidity.

7. Maintenance

- Remove Dirt Particles: Use a soft-bristle brush to gently remove dirt from any spaces between product components. Do not use water for this step.
- Clean Surfaces: Wipe the product surfaces with a damp, non-scratching cloth. Use only cold or lukewarm water.
- Disinfection (if needed): If disinfection is required, use a material specifically suitable for medical devices (risk class I, plastic materials).
- Important: Never put the macu4 medical device in any cleaning machine or appliance (washing machine, dishwasher, microwave, steamer, or similar).

8. Adverse event reporting

Any serious incident related to the device should be reported to the manufacturer and the competent authority of the member state where the user and/or patient is established. Please use the form Please use the Product Experience Report (PER) https://25462115.fs1.hubspotusercontent-eu1.net/hubfs/25462115/mac4_GmbH_Lumo/Product_Experience_Report_EN.pdf and send it to by email to support@macu4.com.

9. Legal notice

Limitation or exclusion of liability

macu4 GmbH, hereinafter referred to as Macu, is only liable if the specified handling and processing instructions, as well as the care instructions and maintenance intervals for the product, are observed. Macu expressly points out that this product may only be used in the component combinations approved by the manufacturer (see instructions for use and catalogues). The manufacturer is not liable for damage caused by component combinations and applications that have not been approved by the manufacturer. Opening and repairing of the products may only be carried out by Macu or with the written permission of Macu.

10. Conformity

The product components of the system Lumo with the Basic UDI-DI marking

Lumo Cuff	426242965P030201RF
Lumo Shovel	426242965P030202RH
Lumo Roll	426242965P030203RK
Lumo Hold	426242965P030204RM

Lumo Clamp	426242965P030205RP
------------	--------------------

meet the general safety and performance requirements according to Regulation (EU)2017/745 for medical devices and the Medical Devices Ordinance (MedDO) of 1 July 2020 (812.213).

Other products

Lumo Socket	customized
Lumo Spacer	customized
Lumo Module Connector	customized

Legal manufacturer

macu4 GmbH, Bücklestrasse 3, 78467 Constance, Germany

CE Marking Details

Macu declares that it complies with the relevant European standards for the design, manufacture and supply of ortho-prosthetic products. The Modules of the Lumo System meet the requirements of the European Regulation (EU)2017/745 Annex I for medical devices. Based on the classification criteria according to Annex VIII of this regulation, the product has been classified as Class I "Accessories". The declaration of conformity has, therefore, been drawn up by the manufacturer under his sole responsibility in accordance with Annex IV of the Regulation. All individual Modules of the Lumo System are marked accordingly.

Trademark

All designations mentioned in this document are subject without restriction to the provisions of the applicable trademark law and the rights of the respective owners. All trademarks, trade names, or company names mentioned herein may be registered trademarks and are subject to the rights of their respective owners. The absence of explicit identification of the trademarks used in this document does not imply that a designation is free of third-party rights.

11. Symbols

The table below shows all symbols (according to ISO 15223-1:2021) used on the product and on the label.

CE Mark

This symbol indicates that a manufacturer declares that the product meets all the legal requirements for CE marking and can be sold throughout the EEA.

Instructions for use.

This symbol indicates that the user must consult the instructions for use.

Manufacturer

This symbol indicates the manufacturer of the medical device.

Warnings

This symbol indicates that caution is required when operating the product or that the current situation requires the operator's attention or action to avoid undesirable consequences.

Temperature limit

This symbol indicates the temperature limits to which the medical device can be safely exposed.

Date of manufacture

This symbol indicates when the medical device was manufactured.

Batch designation

This symbol indicates the manufacturer's batch code so that the batch or lot can be identified.

Catalogue number

This symbol indicates the manufacturer's catalogue number so that the item can be identified.

Medical device

This symbol indicates that the item is a medical device.

Clear device identification

This symbol identifies a carrier that contains information for unique device identification.

Keep away from sunlight

This symbol indicates a medical device that must be protected from light sources.


Magnetic fields

This symbol indicates that magnets are contained in the product. The magnetic field of magnets may affect or disable the function of medical devices (e.g. pacemakers or defibrillators).


12. Technical data

12.1. Cuff

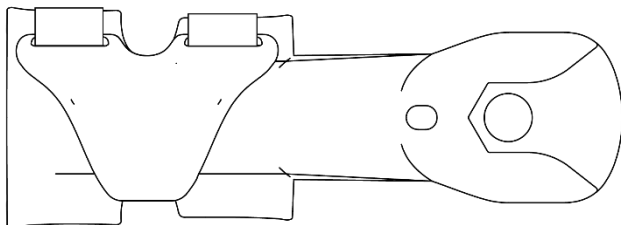
General	
Drawing	 Different versions are available. The present drawing is only informative.
Dimensions (size: STD-SML)	STD-SML-A : 123.8 mm x 56.8 mm x 46.2 mm STD-SML-B : 143.8 mm x 56.8 mm x 46.2 mm STD-SML-C : 163.8 mm x 56.8 mm x 46.2 mm STD-SML-D : 183.8 mm x 56.8 mm x 46.2 mm STD-SML-E : 203.8 mm x 56.8 mm x 46.2 mm
Dimensions (size: STD-MED)	STD-MED-A : 123.8 mm x 62.4 mm x 51.4 mm

	STD-MED-B : 143.8 mm x 62.4 mm x 51.4 mm STD-MED-C : 163.8 mm x 62.4 mm x 51.4 mm STD-MED-D : 183.8 mm x 62.4 mm x 51.4 mm STD-MED-E : 203.8 mm x 62.4 mm x 51.4 mm
Weight	Depending on product size, between 47g and 70g
Material	Cuff shell: PA2200, coloured with DM standard colour (DyeMansion) Protective material: Alcantara, Velour Velcro: Nylon Magnets: Neodymium
Lifetime of the product	2 years at an average of 120 minutes per day with daily use

12.2. Socket

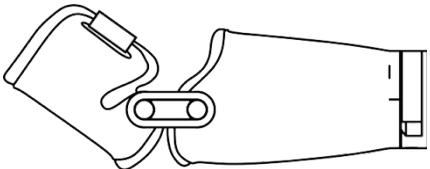
General	
Drawing	
Dimensions	Varies according to product size
Weight	Depending on product size, between 60g and 120g
Material	Module body: PA2200, coloured with DM standard colour (DyeMansion)
Lifetime of the product	2 years at an average of 120 minutes per day with daily use

12.3. Clamp Module

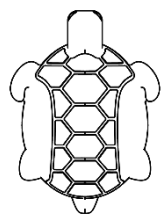General	
Drawing	
Dimensions	51.9 mm x 70.6 mm x 36.9 mm
Weight	23g

Allowed thickness of objects	From 0.15 mm to 5 mm
Material	Module body: PA2200, coloured with DM standard colour (DyeMansion)
Lifetime of the product	2 years at an average of 120 minutes per day with daily use


12.4. Shovel Module

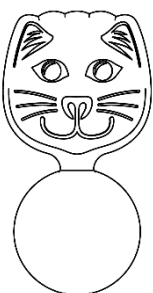General	
Drawing	
Dimensions	51.6 mm x 98.8 mm x 20.6 mm Dimensions may vary depending on the orientation of the Module

Weight	20g
Material	Module body: PA2200, coloured with DM standard colour (DyeMansion)
Lifetime of the product	2 years at an average of 120 minutes per day with daily use

Maximum load forces in the Module	
Maximal Pushing force	30 N
Maximum mass allowed in the shovel	0.5 kg


12.5. Roll Module

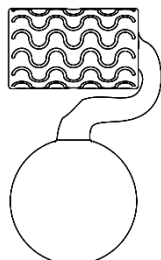General	
Drawing	

Dimensions	53.1 mm x 89.3 mm x 28.0 mm Dimensions may vary depending on the orientation of the Module
Weight	32g
Material	Module body: PA2200, coloured with DM standard colour (DyeMansion)
Lifetime of the product	2 years at an average of 120 minutes per day with daily use

Maximum load forces in the Module	
Maximal Pushing force	30 N

12.6. Hold Module

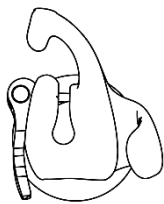General	
Drawing	
Dimensions	52.2 mm x 67.3 mm x 41.1 mm Dimensions may vary depending on the orientation of the Module
Allowed diameters for objects	Maximum 15.00mm *
Weight	50 g
Material	Module body: PA2200, coloured with DM standard colour (DyeMansion) Module head: Ultrasint TPU01
Lifetime of the Module	2 years at an average of 120 minutes per day / daily use

Maximum conditions the Module can withstand	
Maximum object weight	0.5 kg

12.7. Environmental conditions

Environmental conditions	
Storage and transport in the original packaging	0 °C/32 °F to +35 °C/95 °F max. 70 % relative humidity
Storage and transport without original packaging	

Operation	
Storage	0 °C/32 °F to +35 °C/95 °F max. 70 % relative humidity and protect from direct sunlight.

13. Configuration of the System

Please see the measuring instructions on the macu4 website under the heading Lumo System.

14. Returns

If you believe there is a problem with your product, please first read this manual carefully before contacting support@macu4.com or your prosthesis manufacturer (unless you purchased the product directly from Macu). Please use the Product Experience Report (PER): https://25462115.fs1.hubspotusercontent-eu1.net/hubfs/25462115/mac4_GmbH_Lumo/Product_Experience_Report_EN.pdf

Please provide the UDI-DI (starting with "(01) ____") and UDI-PI (starting with "(10) ____") of your Module when requesting warranty repairs or returns. These can be found imprinted on the Module.

If you return your product to us or your prosthetist, please pack it appropriately; the original packaging in which you received your Module is perfectly suitable for this purpose.

15. Disposal of products

If you are no longer going to use a product, you can dispose of it as follows:

- Return to macu4 GmbH
- See FAQs on the website Website www.macu4.com

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- Vrnitev na macu4 GmbH